


BD Nexiva™

Closed IV catheter system-single port
with MaxZero™ needle-free connector

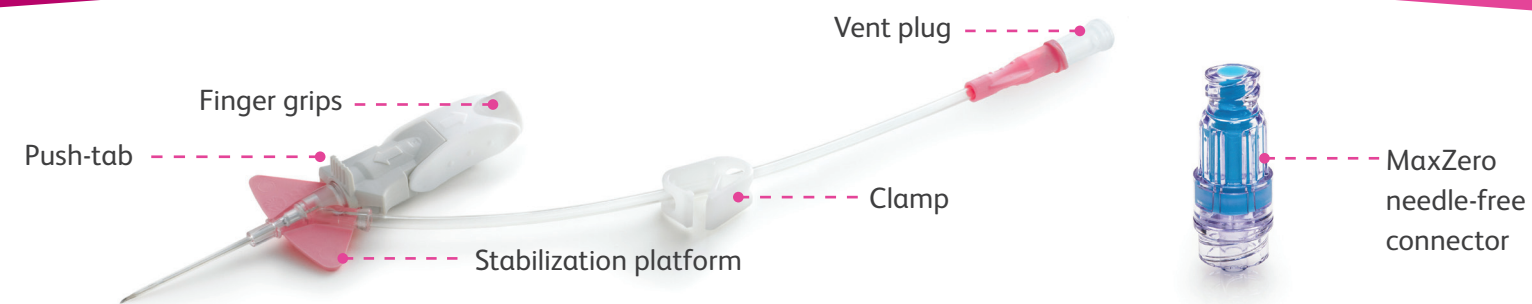

Points to practice*



Preparation

- Prime the MaxZero needle-free connector in the inverted position. Tap while priming to purge air. Return the attached devices to the preparation area
- Secure vent plug
- Clamp should not be engaged
- Twist to remove needle cover
- Holding as shown, pull back approximately 1/8" on finger grips
- Push finger grips back to their original position (close white/grey components) so the stabilization platform and finger grips are snugly together



Insertion and flashback

- Hold the system as shown and access the vessel
- Initial blood return is along the catheter, then up the extension set
- Look at catheter for initial blood return
- Lower and advance the entire catheter and needle unit slightly to ensure the catheter tip is within the vessel



Advancement

- Place pad of index finger behind the push-tab and push the catheter off the needle into the vessel
- Tip:** Do not pull back on the needle during advancement.

Needle removal

- Stabilize the system and pull back until the push-tab component releases from the stabilization platform
- Discard the shielded needle into a puncture-resistant, leak-proof sharps container


Preparation for use

- Engage the clamp
- Remove the vent plug



Access and flushing

- Remove protective cover of the MaxZero needle-free connector and insert the male luer to the catheter hub using a straight on approach
- Release the clamp and flush
- To disconnect, hold the connector and syringe, and with a controlled rotation of the syringe, use a full 360° turn to release the luer lock. Clamp when not in use

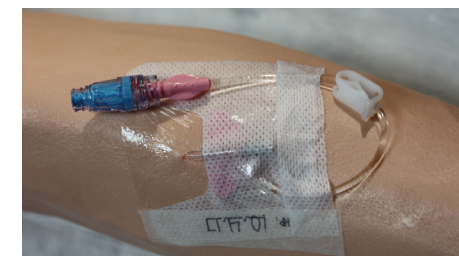

Stabilization

- Apply a transparent dressing to cover the septum, allowing maximum use of extension tubing

*Refer to package insert for complete instructions for use

bd.com

© 2018 BD. BD, the BD Logo and all other trademarks are property of Becton, Dickinson and Company. BD-2750CE (02/18)

