

JOBST[®] FarrowWrap[®] Toe Cap

The JOBST[®] FarrowWrap[®] Toe Cap is for mild to moderate lymphedema patients who want a cost-effective, ready-to-wear option. The Toe Cap works well in combination with the FarrowWrap[®] Footpiece or by itself.



- Suitable for 24-hour use
- Latex-free fabric
- Can be used on either foot
- Trimmable
- Available in 15-20 and 20-30 mmHg*
- Tan & Black colors




DONNING:

- 1. Make sure seams on Toe Cap face outward.
- 2. Insert toes and foot into the toe cap opening.
- 3. Position the Toe Cap so that it comfortably fits over the toes and foot.
- 4. Work Toe Cap in between your toes to web space.
- 5. If the toe lengths are too long, mark the toe at the appropriate length, remove the Toe Cap, then trim the length of the Toe Cap to the marked area and reapply. Be careful not to remove too much of the toe length.

Ordering information

Color	Compression Class	X-Small	Small	Medium	Large
Tan	CCL1 (15-20 mmHg)*	7102700	7102701	7102702	7102703
Tan	CCL2 (20-30 mmHg)*	7294700	7294701	7294702	7294703
Black	CCL1 (15-20 mmHg)*	7102704	7102705	7102706	7102707
Black	CCL2 (20-30 mmHg)*	7294704	7294705	7294706	7294707

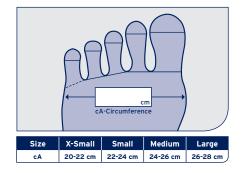

* The mean compression for an average foot size with the related measurements.

Please refer to the product label and / or package insert for full instructions on the safe use of these products.



JOBST[®]. an Essity brand /JOBSTUSA @JOBSTforUSA @JOBST USA

jobst-usa.com

BSN medical Inc., an Essity company 5825 Carnegie Blvd., Charlotte, NC 28209-4633 Tel. (+1) 704 554 9933 Fax (+1) 800 835 4325 To order toll-free: Tel. (+1) 877 417 5187