

QUANTUM 400 SERIES INTERSEGMENTAL TABLES



TABLE MODEL OPTIONS:

- AMQ400 -STANDARD
- AMQ401 -WITH FACE SLOT (UNDER FOOT COVER)
- AMQ400H HEAT
- AMQ400P- PATIENT ASSIST BAR
- AVAILABLE WITH OPPOSITE SIDE CONTROLS

ALL OPTIONS INCLUDE:

- PROTECTIVE COVER FOR FOOT AREA
- BOLSTER KNEE PILLOW (20"L x 6"W x 7"H)





AMQ400 SERIES STANDARD FEATURES:

- MOTORIZED ROLLER ELEVATION AND VIBRATION
- HAND HELD REMOTE CONTROL WITH CORD
- REMEMBERS LAST TREATMENT SETTINGS
- LOCK FEATURE PREVENTS PATIENT FROM CHANGING SETTINGS
- ULTRA SMOOTH NYLON MASSAGE ROLLERS
- ROLLER PRESSURE IS ADJUSTABLE WITH LED DISPLAY
- VIBRATION IS SELECTABLE (0-30 MINUTE TIMER)
- FOUR BALL CASTERS













OPERATION OF CONTROLS



POWER—Turns the main power to the table on or off.



VIBRATION—Turns the vibration mode on or off.



HEAT- (available on Model AMQ400H) Heats up to 108°—110° F in approximately 6 minutes.



DIRECTION– Determines the counter-clockwise or clockwise direction of the rollers when in motion. Standard position is counter-clockwise rotation.



LOCK- Prevents patients from changing the table settings. To unlock, hold in button for three seconds.



NOTE: Remote control automatically defaults to last settings used.



TREATMENT TIME INDICATOR



START/STOP – Push once to start selected therapy. Give to the patient to use as a remote stop.



TRAVEL – Controls the tracking of the rollers from cervical to thoracic to lumbar. The ON position is when the button is lighted, which allows the rollers to track the full range. The OFF position is when the button is not lighted in which the horizontal movement of the rollers is halted and allows the treatment to concentrate on selected area. The travel of the roller carriage is 18".



TIMER/ROLLER HEIGHT CONTROL—Timer may be set from 1 to 30 minutes by using the arrow buttons. After the timer has been set, toggle up or down to set the height of the rollers. The rollers will retract to the lowest position when time expires.



ROLLER LOCATION INDICATOR PANEL

Red light behind the Cervical-Thoracic-Lumbar panel indicates horizontal positions of rollers.



