

FORM FIT™ CERVICAL COLLAR


Clinical Application Instructions

BEFORE USING THIS MEDICAL DEVICE, PLEASE READ THE **INSTRUCTIONS FOR USE** COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE. THIS PRODUCT IS INTENDED TO BE USED UNDER THE SUPERVISION OF A HEALTHCARE PROFESSIONAL.

IDEAL FOR: Post-operative rehabilitation, cervical sprains and strains.

SIZING:

- 1** Measure the height from the sternal notch to the chin on patient. (figure A)
- 2** Next, measure the circumference of the patient's neck. (figure B)
See chart at right for size selection.

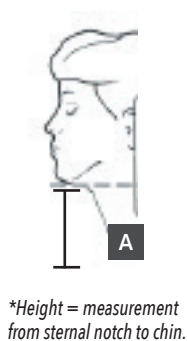

H* X L	CIRCUM.	SIZE
2" x 14"	7" - 12"	Ped./2XS
2" x 16.5"	9" - 13"	XS
3" x 18.5"	11" - 16"	S
3" x 22"	15" - 20"	S/Long
4" x 20"	13" - 18"	M
4" x 23"	16" - 21"	M/Long
4.5" x 22.5"	15" - 20"	L
4.5" x 24"	16" - 22"	XL
2" x 24"	17" - 22"	Long/Nar.

APPLICATION:

- 1** Center the cervical collar under the chin. (Figure 1)
- 2** Wrap both ends around the back of the neck and secure hook and loop closures. (Figure 2)
Adjust for comfort, ensure collar doesn't interfere with normal breathing.

CLEANING INSTRUCTIONS:

Hand wash in cold water using mild detergent. Rinse thoroughly and air dry.

MATERIAL CONTENTS:

NOT MADE WITH NATURAL RUBBER LATEX.

WARRANTY:

DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale

WARNINGS AND PRECAUTIONS:

If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

Rx ONLY.

INTENDED FOR SINGLE PATIENT USE.

