

Safe·t mate[®]

Instructions for configuring Safe t mate Model SM2-3 to prevent chair from rolling back when user is seated.

This modification removes the seat lever that disengages the brakes arms in reaction to the user being seated in the wheelchair. Therefore, this results in the wheelchair still being able to move forward but it will prevent rearward mobility. The only way an occupied chair in this configuration can be moved backward is for the caregiver to lift the temporary override lever to release the brake arms from the tires.

CAUTION

**Please be advised that if the wheelchair user rolls up to a wall, desk, bed, window, etc.,
*they will not be able to back up.***

For this reason, this configuration should only be used in a highly supervised environment.

The use of rear anti-tippers (Safe t mate Model SM-008 or similar)
is strongly recommended in combination with this configuration.

1. Install SM2-3 Anti-rollback Device Completely per instructions.
2. Remove Seat Lever Assembly consisting of lever and articulating seat pad (highlighted in gray). Retain 2 bolts and nuts along with assembly for future use

