

Safe•t mate

Product use and specification sheet

Model Number	SM-004B
Brand Name	Safe•t mate ®
Description	Personal Fall Monitor
Recommended Use	The Safe•t mate Personal Fall Monitor is ideal for long-term care residents confined to a wheelchair or bed who are at risk for falls and need a staff alert as an alternative to a restraint.
Not recommended for	Patients who need a restraint Patients who may become agitated by the feeling of the garment clip attached to their clothing.
Operation	Alarm box does not have an on/off switch. The alarm will sound at the moment the magnet is pulled away from the metal face plate and can only be silenced by re-applying the magnet. The Cord features a slide adjustment. There should be enough slack in the cord to allow for normal movement without accidentally releasing the magnet.
Specifications	Alarm box ABS Plastic with Silicone sleeve. Powered by 9 volt battery. 95 db horn volume. LED indicator normally off. Flashing = low battery Low power drain circuit. Recommended battery change = Monthly Magnet assembly 1.4 MGOE low energy rubber magnet with energy field reaching < 1" from magnet Nylon cord length adjustable from 18" – 35"
Care	Wipe unit with cloth and mild spray cleaner such as glass cleaner. Do not submerge. Replacement Magnet assembly - Part no. 004BMAG