gazelle ps

Gazelle PS stands for Prone and Supine. It means the Gazelle PS can be used by children in a posterior or anterior position



The many adjustment possibilities and the wide range of accessories mean that it is adjustable to children with different types of disabilities.

The standing position is very important for both the physical and psychological well-being. When the child is placed in the prone position, the maximum stretch of both hip and knee is obtained.

Gazelle PS is available in three sizes and is suitable for children from 1 to 14 years of age. Not all children are alike - that is why it is important to use the measuring chart to find the correct size for the child.







O to 30°

max. 176lb



19-26"



30-36'



Tilt vertical-horizontal

The Gazelle PS tilts from vertical to $15\,^\circ$ past horizontal. This is an excellent position for respiratory therapy and postural drainage



Manual leg abduction

Moves in an anatomically correct motion to help seat the head of the femur into the acetabulum. You can adjust either one or both legs to the optimal angle



Foldable

From the upright position the Gazelle PS is easily folded. By loosening only one screw, abducting the legs and by lifting it slightly, the Gazelle PS is folded for storage or transportation



Multi-adjustable foot supports

Ball joints in foot plates allow for positioning and support for almost any ankle/foot placement

Standard

As standard the Gazelle PS is supplied with chest and foot straps. Please order accessories from the following pages



Frame color

Delivered in different colors for size 1, 2 and 3



Yellow for size 1



Green for size 2



Blue for size 3

Tilt vertical-horizontal

The tilt angle can vary from child to child. Tilt angle from 90 $^{\circ}$ to -15 $^{\circ}$



tilt 90°




tilt 45°



tilt 0°



keep track of the tilt angle with the protractor

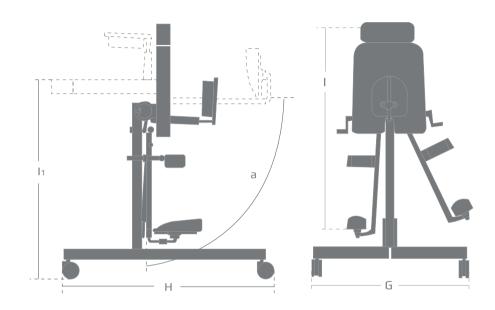

Measurements

	Gazelle PS Size 1	Gazelle PS Size 2	Gazelle PS Size 3
Width, total (G)	48 cm (19")	58 cm (23")	65 cm (26")
Length total (H)	76 cm (30")	88 cm (35")	99 cm (39")
Length, foot-axila (I)	45-95 cm (18-37")	60-105 cm (24-41")	70-125 cm (28-49")
Height, when horizontal (I1)	76 cm (30")	80 cm (31½")	90 cm (35")
Height, folded	28½ cm (11")	29½ cm (12")	29½ cm (51")
Length, folded	95 cm (37")	113 cm (44½")	130 cm (51")
Angle, board (a)	90° to -15°	90° to -15°	90° to -15°
Weight	20 kg (44 lb)	22 kg (48 lb)	24 kg (53 lb)
Max. load/user weight	40 kg (88 lb)	60 kg (132 lb)	80 kg (176 lb)

ACCESSORIES



Head support system

88124-xx. For correct positioning and support in the vertical or prone position

one size



Mounting for head support

88130

For mounting of other head supports. See accessories section

one size



Head support with facial aperture

88117-x

For correct position and support

size	measurements
1	H:81/4 x W:61/4"
2	H:10 x W:81/4"
3	H:11½ x W:10¾"



Side supports

88055-x

For additional support and safety

size	measurements
5	H:5¾ x W:5¾"
L	H:7 x W:5"



Anat.head support with facial aperture

88119-x

For correct positon and support

size	measurements	
1	H:10 x W:11"	
2+3	H:103/4 x W:12"	



Side supports, interior

88075-x

For additional support and safety

size	measurements
5	H:5¾ x W:5¾"
L	H:7 x W:5"



Trunk support

88106-x For extension of the trunk support. Helps support the child's trunk position

size	measurements
1	H:9¾ x W:9¾"
2	H:4 x W:11"
3	H:4 x W:121/2"



Side supports, exterior

88065-x For additional support and safety

size	measurements
5	H:5¾ x W:5¾"
L	H:7 x W:5"



Angling of lower leg

88165-x

When individual angling of the user's lower legs is needed



Strap for side supports

88250-x

For added safety

size	measurements
1	L:15¾ + 15¾ end strap
2	L:19¾ + 17" end strap
3	L:23¼" + 17" end strap



Tray

88112-x or 88114-x Available as standard or with extended arm rests

size	measurements
1	W:23½ x D:21½/D:29½"
2	W:23½ x D:21/D:29½"
3	W:23½ x D:21½/D:29½"



Tray with bowl

88111-xN or 88113-x Available as standard or with extended arm rests. The cover plate follows as standard

size	measurements
1	W:23½ x D:21½/D:29½′
2	W:23½ x D:21/D:29½"
3	W:23½ x D:21½/D:29½"

ACCESSORIES



Tray88108-0
Can be equipped with armrests
861131, Toybar 861135 and Grip
handle 861136

size measurements one W:24 x D:22½"

