





# How to measure the user


This is a simple guide to make sure the chair is correctly configured for the user. To measure accurately, always make sure that the user is lying on a firm flat surface with hips, knees and ankles at a 90 degree angle.

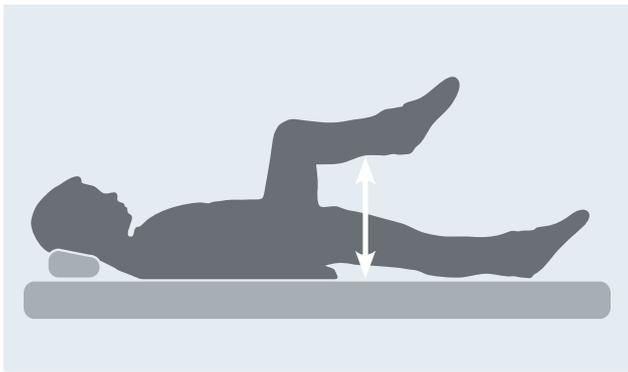


Seat depth

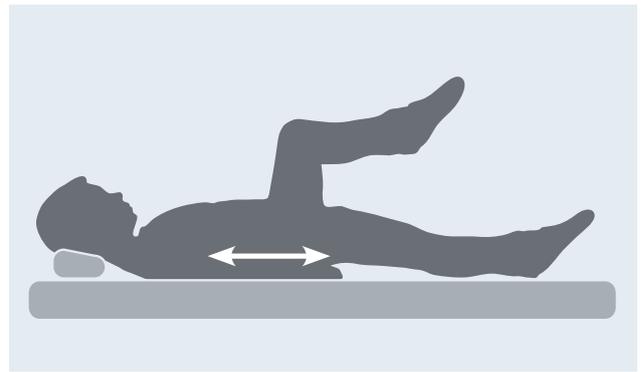


Armpit height

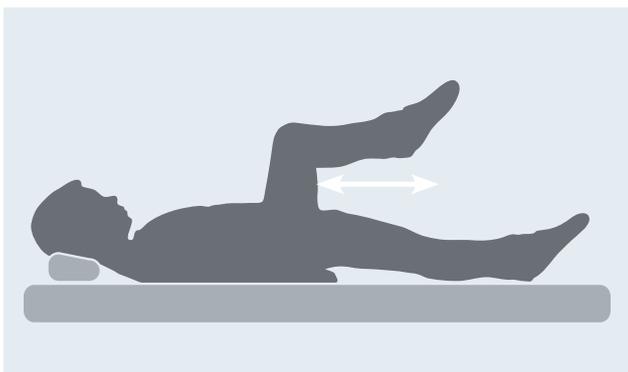

Lower leg length

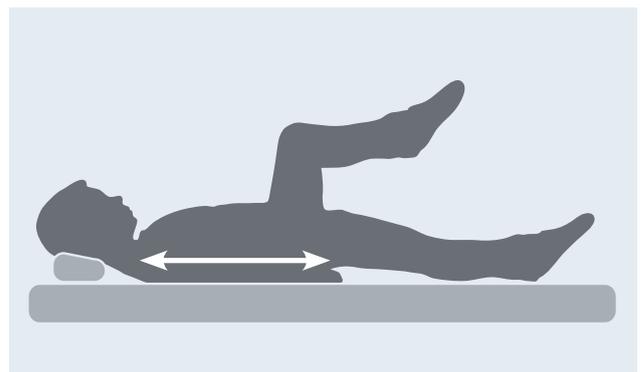


Back height

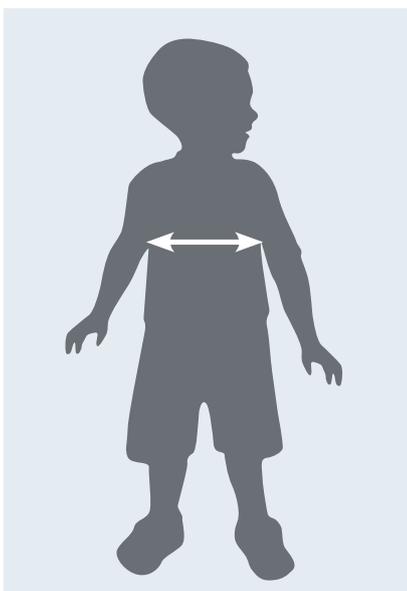

Chest width




Wombat Living is designed to adjust for growth and changing needs. This picture shows how the seat can be altered and illustrates how important good measurement is.

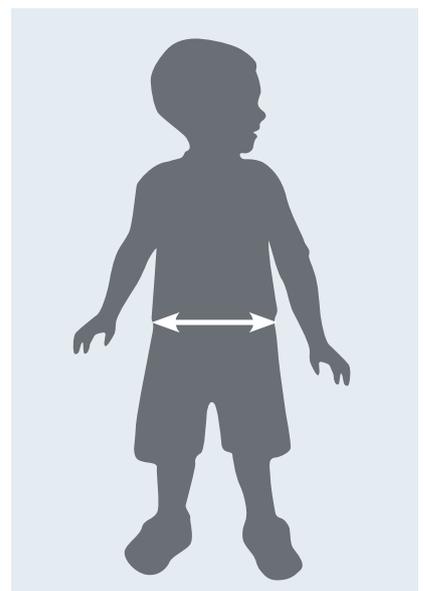

Hip width

# Measurements

Wombat Living has built-in growth, meaning adjustable seat depth and hip supports to adjust seat width. Once you have measured the user, you can easily identify the correct chair size in below table.

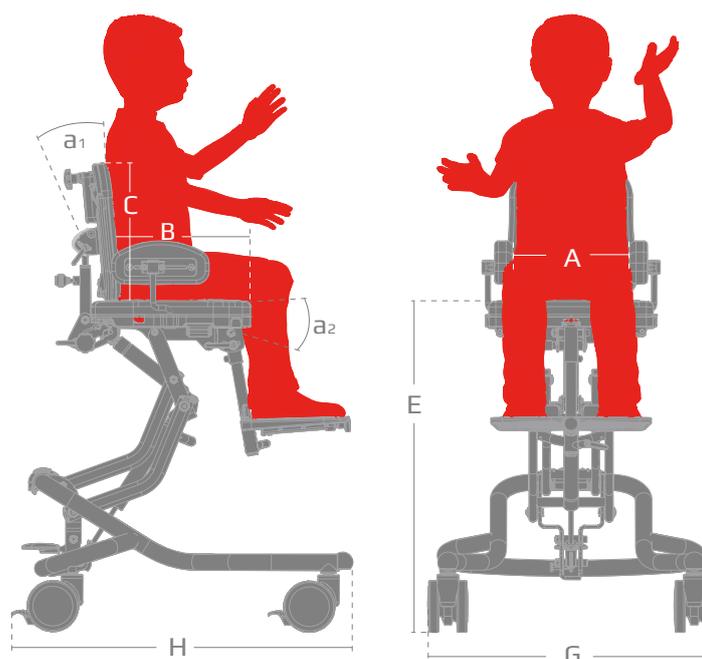

## Measurements

	Wombat Living size 1	Wombat Living size 2	Wombat Living size 3
Seat width (A)	30 cm (11¾")	37 cm (14½")	42 cm (16½")
Seat width between hip supports (accessories)	21-30 cm (8¼-11¾")	28-37 cm (11-14½")	25-42 cm (9¾-17¼")
Seat depth (B)	18-31 cm (7-12¼")	25-38 cm (9¾-15")	32-50 cm (12½-19¾")
Back height (C)	28-34 cm (11-13½")	31-38 cm (12¼-15")	39-50 cm (15¼-19¾")
Lower leg length (D)	15-25 cm (6-10")	15-35 cm (6-13¾")	15-44 cm (6-17¼")
Seat height above floor, 75 mm castors (E)	36-65 cm (14¼-25½")	36-65 cm (14¼-25½")	39-67 cm (15¼-26½")
Seat height above floor, 100 mm castors (E)	39-68 cm (15¼-26¾")	39-68 cm (15¼-26½")	42-70 cm (16½-27½")
Back width, flat cushion (F)	25 cm (9¾")	30 cm (11¾")	35 cm (13")
Back width, contoured cushion, between side supports (F)	17 cm (6¾")	23 cm (9")	27 cm (10½")
Arm rest height above seat cushion	6-17 cm (2¼-6¾")	6-17 cm (2¼-6¾")	6-17 cm (2¼-6¾")
Back angle (a1)	90° to 130°	90° to 130°	90° to 133°
Seat angle (a2)	-15° to 20°	-15° to 20°	-10° to 22°
Angle, foot support bar (a3)	-30° to 30°	-30° to 30°	-30° to 30°
Frame width (G)	55 cm (21½")	55 cm (21½")	61 cm (23¾")
Frame length (H)	60 cm (23½")	60 cm (23½")	72 cm (28")
Max. user weight / load	45 kg (99 lb)	45 kg (99 lb)	70 kg (154 lb)
Weight	15,5 kg (34 lb)	15,5 kg (34 lb)	22 kg (48.5 lb)