



## QUICK START GUIDE

# BIG Candy Corn™ & LITTLE Candy Corn™ 2

accessibility switch

---

### WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full

AbleNet, Inc.



Made with recycled content

EN July 26, 2021 8:52 AM



## Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

### To get started:

1. Connect switch plug to a device with a 3.5-mm / 1/8-in TS (mono) jack.
2. Turn switch on.
3. Switch will activate when a user is within 10-mm / 0.4-in of or touching the active proximity sensor area.

*This device can do more!*



ableCARE™

Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

### 3 Device Overview

