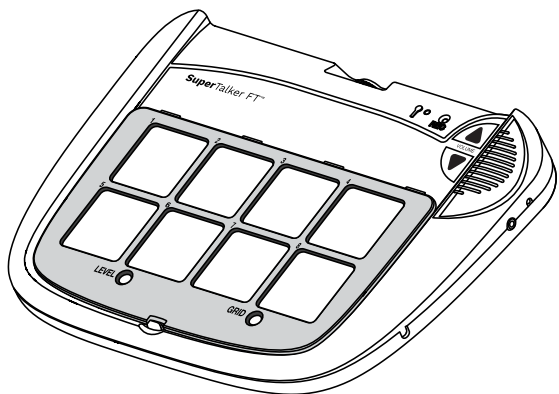
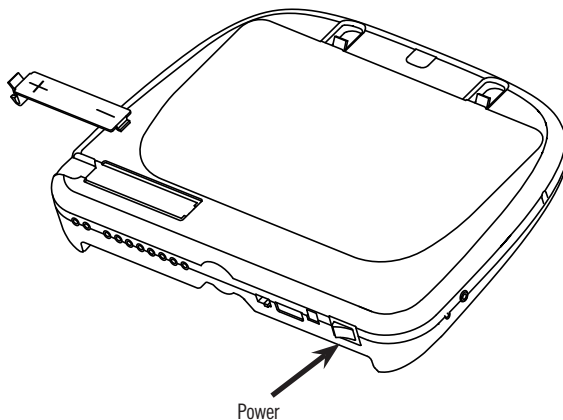
# Quickstart Guide: SuperTalker™ FT progressive communicator

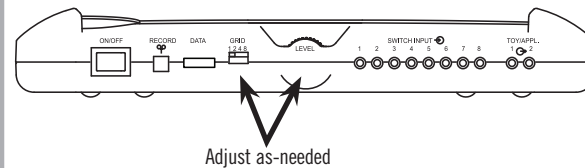
AbleNet®

## 1 Powering SuperTalker

Insert four AA batteries into the device as shown. Power On/Off is located on the spine of the device.

## 2 Select Level and Grid


With the desired Overlay and Keyguard in place:  
Select a LEVEL (1-8) and Grid (1, 2, 4, 8) using the controls on the spine of the device.

*NOTE: Be sure to match the Grid number to the Keyguard and Overlay currently installed on the device.*


*For information on Overlays, Keyguards and Helpful Tips, see reverse.*

## 3 Enter Record Mode

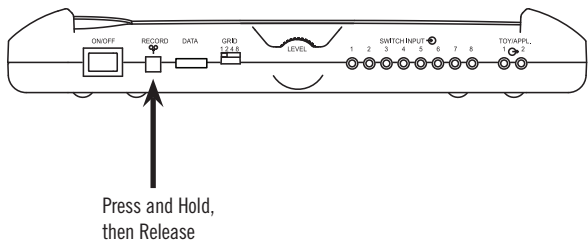Press and Hold the RECORD button until the RECORD Light activates, then Release.

## 4 Record Messages

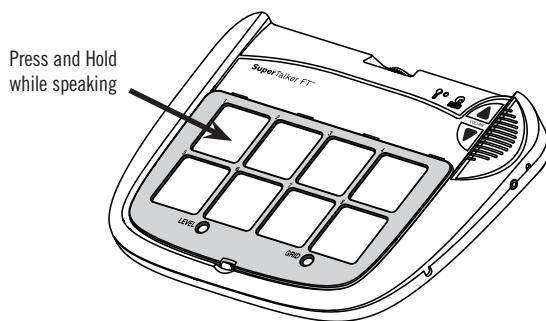
Press and Hold the desired Message location, begin speaking when you hear a beep. Release when finished.

Repeat for additional messages.

RECORD Light will flash while recording.

*NOTE: Speak close to the microphone for loud, clear sound.*


## 5 Exit Record Mode

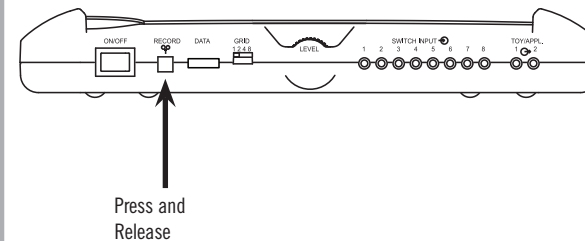Press and Release the RECORD button to exit Record Mode.

*NOTE: This device has 16 minutes of total recording time. The RECORD Light will turn yellow when only two minutes remain.*

## Advanced Features
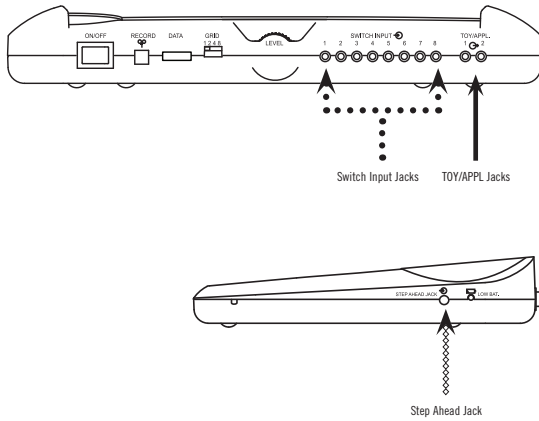
### Playing a message with an External Switch

Attach the switch of your choice with a 1/8-inch (3.5mm) to the Switch Input jacks and continue to use the device as before. (Jacks indicated with .....)

*NOTE: Switch Input jacks are mapped to message locations 1 - 8 from left to right, depending on the grid used.*

### Controlling Toys or Appliances

Plug any switch-adapted toy or appliance into the TOY/APPL jack. When the SuperTalker is activated, the attached toy or appliance will activate for the length of the recorded message. (Jacks indicated with \_\_\_\_\_)

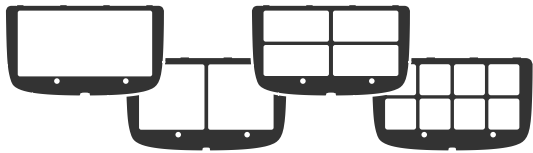
*NOTE: TOY/APPL jacks are mapped to the first two message locations on the SuperTalker. To control an electrical appliance, a PowerLink® 4 is required.*

### Using the Step Ahead jack

Plug the switch of your choice into the Step Ahead jack to access the messages, in-order from location 1 - 8. (Jacks indicated with .....)

*NOTE: Step Ahead jack will move through the message locations based on the Grid selected.*

## Keyguards and Overlays

### About Keyguards

SuperTalker includes keyguards in 1, 2, 4, and 8-grid formats. Release the installed keyguard by pressing on the tab holding the bottom center of the keyguard. Small numbers at the top of each message location correspond to the External Switch jacks (see above).

### Keyguard Storage

The keyguard storage area on the back of SuperTalker can hold up to three keyguards at once. Plain paper overlays may be stored here as well.

## Helpful Hints

### Recording Tips

While in Record Mode, if the SuperTalker is not activated after 20 seconds, the device will exit Record Mode.

Master Clear - Press and hold the RECORD button for 10 seconds to delete all messages from SuperTalker.

You can only record one message grid per level.

### Playback Tips

If the Keyguard in-use does not match the Grid selected, SuperTalker will not play the correct message.

If you press a message location where no message has been recorded, the RECORD Light will flash yellow.

### Volume Controls

Lock or Unlock SuperTalker's Volume controls by pressing the bottom left and right message locations simultaneously.

When adjusting volume,

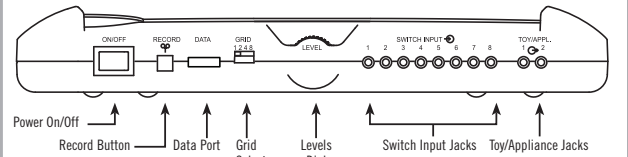
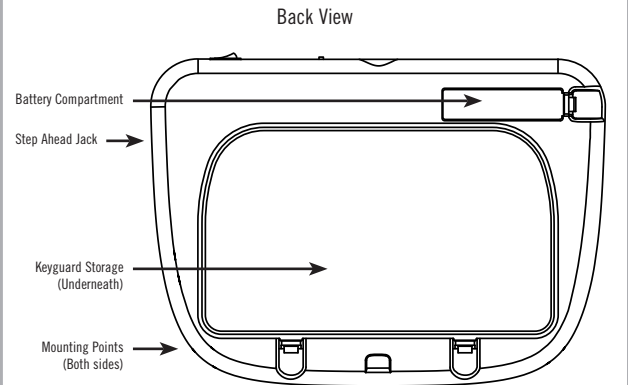
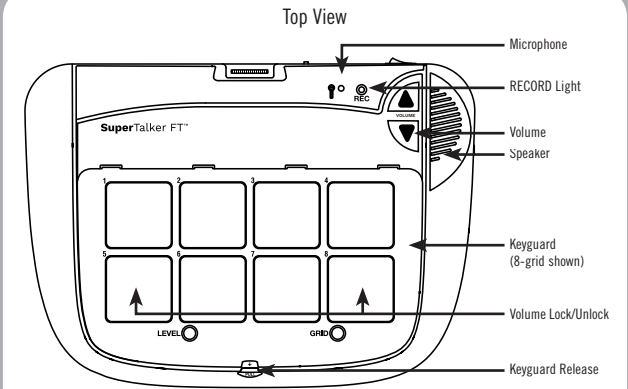
RECORD Light Yellow – Volume unlocked

RECORD Light Red – Volume locked or max/min

### Storing Your Device


Remove batteries before storing device.

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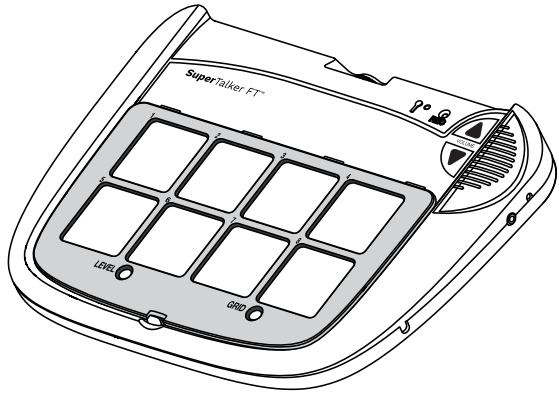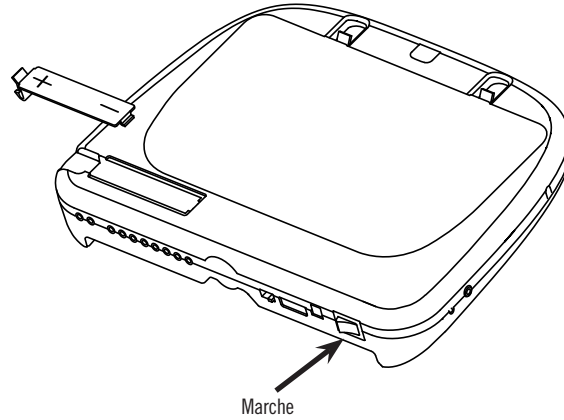
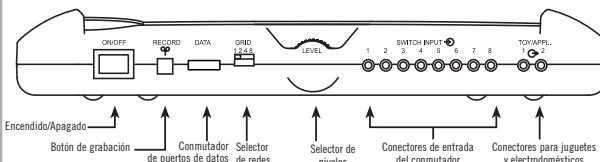
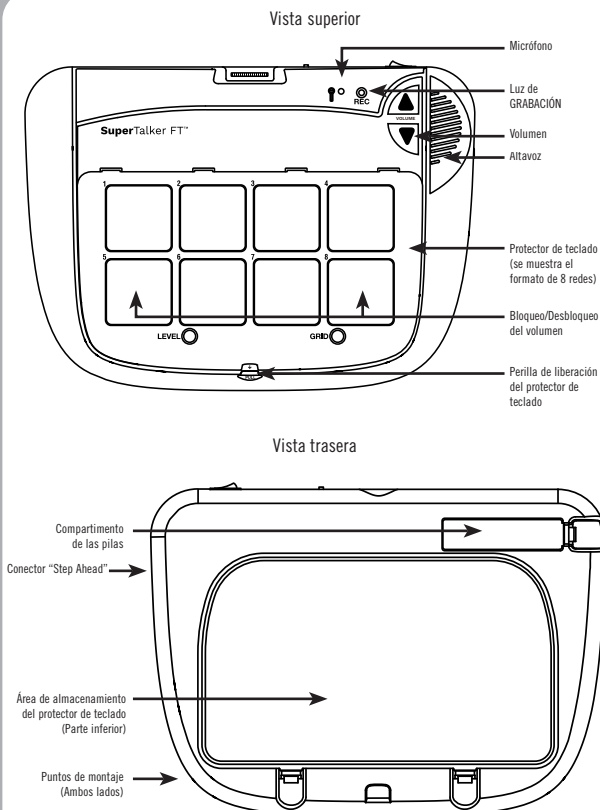
## Warranty & Contact Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information

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## For private households:

Information on Disposal of Users of WEEE

This symbol  on the product and/or accompanying documents means that

used electrical and electronic equipment (WEEE) should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product. to designated collection points where it will be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product.

Disposing of this product correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling. Please contact your local authority for further details of your nearest designated collection point. Penalties may be applicable for incorrect disposal of this waste, in accordance with your national legislation.

## For professional users in the European Union:

If you wish to discard electrical and electronic equipment(EEE), please contact your dealer or supplier for further information.

For disposal in countries outside of the European Union:

This symbol is only valid in the European Union(EU). If you wish to discard this product please contact your local authorities or dealer and ask for the correct method of disposal.

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