

# NEW and IMPROVED!!

**NEW SAFETY FEATURE!!!! ProSeries 1412 , ProSeries 1416, and ProSeries 1420 now have a safety loading brake\* to aid in a safer transfer of rider!**

\*Because of this new safety loading brake the ProSeries 1412, 1416, and 1420 the hand brake is not included. The hand brake will be an optional accessory for the 1416 and 1420 ProSeries Model. The hand brake can be requested by calling.

## Installation Instructions for Loading Brake

- 1) Remove the two Allen bolts from the loading brake extension tube.
- 2) Place the loading brake extension tube over the right rear axle so the handle is pointing towards the front of the AmTryke, making sure the handle is in the down position and over the tire.
- 3) Install the two Allen bolts up through the attached bracket on the right rear axle and then turn them clockwise into the loading brake extension tube.
- 4) Use the double ended Allen wrench provided with your AmTryke to tighten up the two Allen head screws from the bottom.

