



Assembly Instructions

Lumbar Pad

Part Number 30-12-0334

Fits # 30-13-1400 1400 Seatback only


Contents of Lumbar Pad

Lumbar Pad with mounting bracket and 2 knobs


Installation Instructions

- 1) Remove the push bar/ handle from the top of the 1400 seat back by pulling upwards. In more difficult cases a rubber mallet may be needed. If the push bar/ handle is not easy to remove, tap lightly on the underneath with a rubber hammer until handle is worked free.
- 2) Remove existing padded back pad with lateral attachments by turning the two knobs located on the back counter clockwise (Left). Then pull upwards and off.

Turn knob counter clockwise to loosen

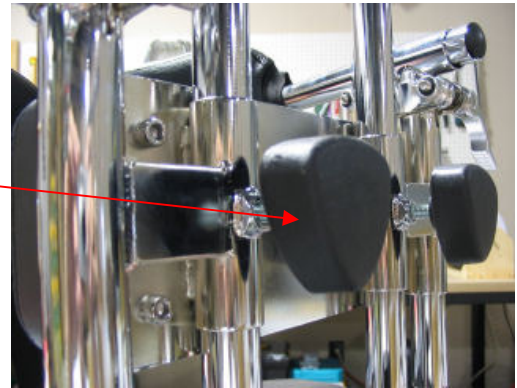

- 3) Loosen the two knobs located on the back of the lumbar pad mounting bracket before trying to install on support tube.

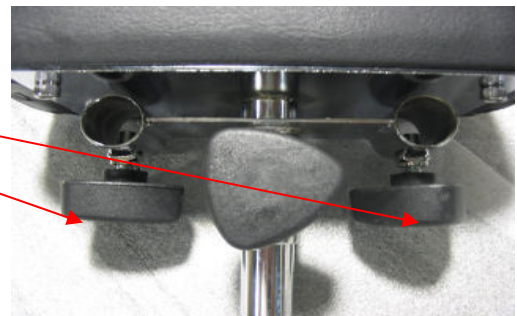

- 4) Slide the lumbar pad down the two support tubes to the desired location.
- 5) Re-install the padded back pad with lateral attachments. Reinstall the handle.

Lumbar Pad Adjustments

The lumbar pad has two adjustment features:

- 1) The position of the lumbar pad can go up and down the support tubes to the desired location.
- 2) The horizontal positioning of the lumbar pad is achieved by loosening the center knob.

NOTE: BEFORE PLACING A CHILD ON THE AMTRYKE AFTER INSTALLING THE LUMBAR PAD CHECK THAT ALL KNOBS ARE TIGHT.