


Attainment Company's

GoTalk[®] 9+




Message Keys activate with the lightest touch!

3 Core Vocabulary Keys – 15 seconds each
9 Message Keys – 10 seconds each on all 5 levels

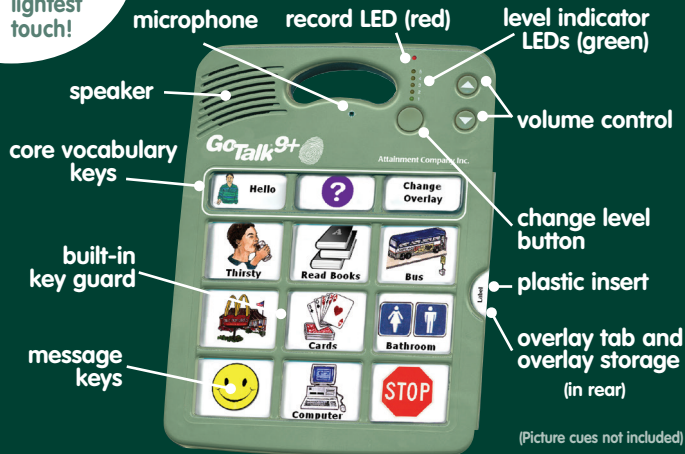

(Picture cues not included)

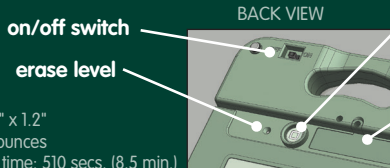

Size: 9" x 12" x 1.2"
Weight: 20 ounces
Total record time: 510 secs. (8.5 min.)



Attainment Company, Inc.



RoHS COMPLIANT

Item # GTL-09

ISBN# 978-1-57861-560-5

Record Message Keys

Turn on the **GoTalk[®] 9+ Life Touch** and you'll hear a beep! Press and release the record button. The level indicator LED (green) and the record indicator LED (red) will light. While they are lit, press and release the message key that you want to record on. As you press the message key, the record LED blinks to indicate it's recording.

When you finish the message, press any key to stop recording. Continue to record each message or record the entire level. When you've recorded all your messages (or the entire level or levels), press and release the record button.

Three Core Vocabulary Keys

Follow the directions above to record a message on the core vocabulary keys. The core vocabulary messages stay the same on each level until you record over them.

Play a Message

Press and release a message key; the level indicator LED will blink.

Levels

To change level, press and release the change level button below the green LEDs. The LED indicating the current level will blink. Press again to change to another level. Change the paper overlay to correspond to the new level.

To erase level, press and release the record button on the back of the device. Push the change level button to the level you want to erase. Insert a pen point or paper clip into the pinhole (erase level) on the back of the unit. This will erase the entire level. Repeat steps above to erase other levels. **Erase level does not erase core vocabulary words or phrases.**

Lock Features (eliminates unwanted use of these buttons)

- **Level lock.** To activate level lock: Press and hold down the record button on the back of the unit. The record LED and level indicator LED will turn solid. While still pressing the record button, press and release the volume down button. The record LED and the level indicator LED will turn off to show you level lock is ON.
- **Record lock.** To activate record lock: Press and hold down the record button on the back of the unit. The record LED and level indicator LED will turn solid. While still pressing the record button, press and release the volume up button. The record LED and level indicator LED will turn off to show the record lock is on.

To **UNDO** the record and level locks, turn the unit off. While off, hold down the record button and turn the unit on. All LEDs will flash briefly, and the unit will beep, indicating that both record lock and level lock are now off.



Designed and Manufactured for Attainment Company, Inc.
by Reimer Electronics Inc., Naples, FL.