



**GoTalk[®]
Select**



GoTalk[®]Go



- Portable
- Excellent sound quality
- Rechargeable

GoTalk® Wearables!

Record and play messages on these wearable, rechargeable AAC devices from Attainment Company.

The **GoTalk® Select** has four message keys, each with three levels. Wear the GoTalk Select with a lanyard or slide-on clip. Magnets on the back attach to any metallic surface.

The **GoTalk® Go** is worn like a watch and features three message keys, each with three levels. Overlays are protected by a removable plastic cover.

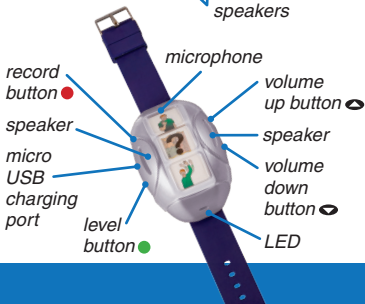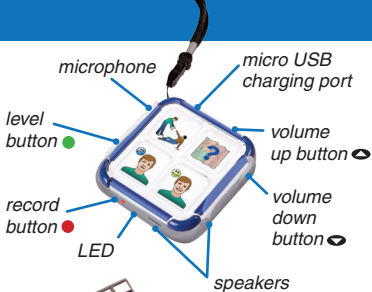

Instructions for Use

Plug in the device to charge for 6 hours before initial use.

Refer to the **GoTalk[®] Go** and **GoTalk[®] Select** layout diagrams.

To turn on: Press and hold the **Volume Up button** until tone is heard.

To turn off: Press and hold the **Volume Down button** until tone is heard.

Record mode on/off: Press and release red **Record button** with a fingernail or closed pen. When in Record mode, a red light is on.

To record a message: Touch a **message key**. A blinking, red light indicates recording is in progress. Record for up to 28 seconds. Touch any key to stop recording.

*NOTE: To check the recorded message without leaving Record Mode, press either **Volume button**.*

To play message: With Record mode off (no red light), touch a **message key** to play the message. While playing messages, **Volume buttons** adjust volume up and down.

To change levels: Press green **Level button** with a fingernail or closed pen. Level 1 beeps once with green light. Level 2 beeps twice with blue light. Level 3 is three beeps with purple light.

NOTE: When changing levels in Record mode, the correct number of beeps is heard for each level, but the light remains red.

To erase all messages:

Press and hold the **Record button** for 10 seconds until tone is heard.

To insert overlays: Remove the protective plastic cover. Add a paper overlay on top of white membrane. Snap cover back on.

To recharge battery: Battery will run for up to 15 hours of continuous use. When battery is low, an orange light will flash towards the end of message playbacks. A micro USB cord is provided for recharging.

NOTE: Routinely recharging the unit is recommended to enhance battery life and usage.

Level Lock: Press and hold the green **Level button** and the **Volume Up button** simultaneously until a tone is heard. This will lock you into the level you are currently on.

To Undo Level Lock: Press and hold the green **Level button** and the **Volume Down button** simultaneously until a tone is heard. You will now be able to change levels.

Record Lock: Press and hold the red **Record button** and the **Volume Up button** simultaneously until a tone

is heard. The record feature is now disabled.

To Undo Record Lock:

Press and hold the red **Record button** and the **Volume Down button** simultaneously until a tone is heard. The record feature is now enabled.

