


Aeromesh™ Calf Strap and Calf Panel


INSTALLATION AND USER'S INSTRUCTIONS

ART NOS. SP102S, SP102M, SP102L, SP103S, SP103M & SP103L

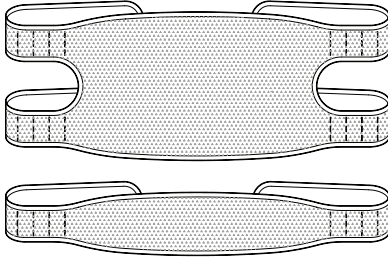

PACKAGE CONTENTS


- Aeromesh™ Calf Panel
- or
- Aeromesh™ Calf Strap
- Adhesive Backed Hook and Loop Patch

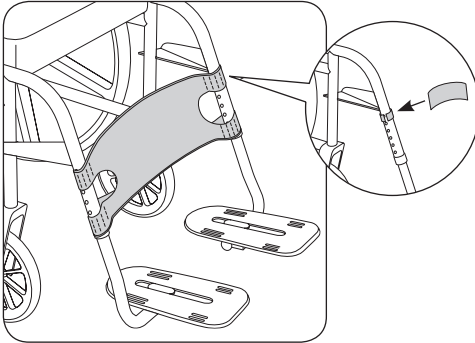

MAINTENANCE: Check periodically for signs of wear in the stitching, webbing, and pads. If significant wear is found, contact your supplier for qualified repair or replacement by Bodypoint. Under no circumstance should this product be altered or repaired by unqualified persons – health and safety depend on it!



CLEANING: Machine wash, warm, 40°C (100°F). Do not bleach. Tumble dry, low temperature, or drip dry. Do not iron. (Placing the product inside a cloth bag during washing helps prevent scratches to the product and the machine.)

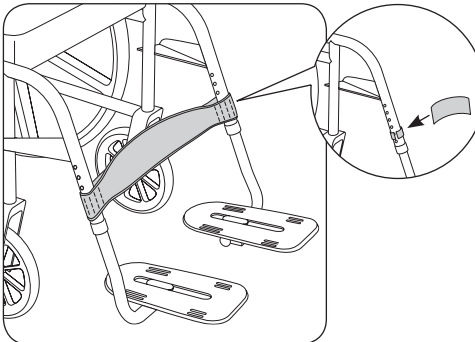

WARRANTY: This product carries a limited lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.

For more information on Bodypoint products, and a list of distributors outside the USA go to www.bodypoint.com.

 Bodypoint®

 EC REP

MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany

 CE