


Sub-ASIS Pad

INSTALLATION AND USER'S INSTRUCTIONS

ART NOS. PD401-2 & PD402-2

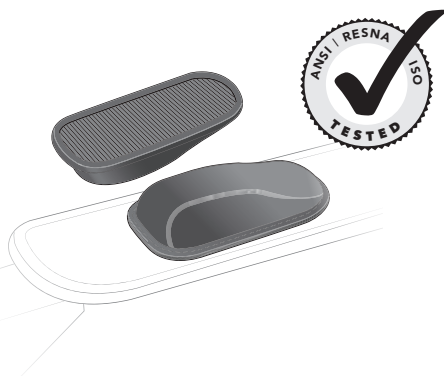


PACKAGE CONTENTS

2 Sub-ASIS pads

This product is intended for use with Art. Nos:

LB235-S38	LB435-S38	LB416-S38
LB205-M46	LB225-M46	LB209-M46
LB229-M46	LB405-M46	LB425-M46
LB425-M46	LB409-M46	LB429-M46
LB205-L62	LB225-L62	LB405-L62
LB425-L62		

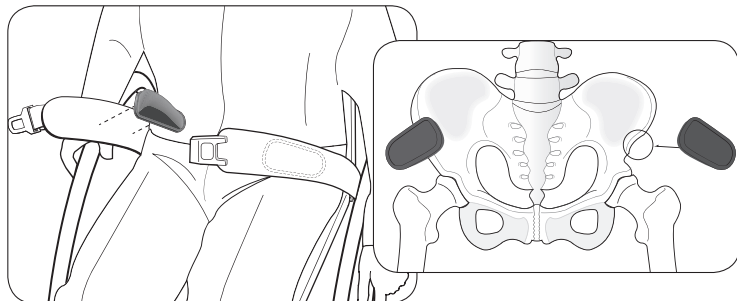

Give these instructions to the user or their caregiver and review them to ensure that they are understood.

⚠️WARNING! Product should be installed and fitted by a qualified rehab technician.

⚠️WARNING! As with any new seating support, this product may change the way a person sits. Users must continue to practice regular pressure relief activities and skin integrity checks, not only where this product contacts the user, but also in primary pressure-bearing areas such as the sacrum, legs, and buttocks. If increased skin redness or irritation occurs, discontinue use and consult your physician or seating specialist. Failure to do so may result in serious injury, such as pressure ulcers.

MAINTENANCE: Check periodically for signs of wear in the stitching, webbing, and pads. If significant wear

is found, contact your supplier for qualified repair or replacement by Bodypoint. Under no circumstance should this product be altered or repaired by unqualified persons – health and safety depend on it!



CLEANING: Machine wash, hot, 60°C (140°F). Do not bleach. Tumble dry, low temperature, or drip dry. Do not iron. (Placing the product inside a cloth bag during washing helps prevent scratches to the product and the machine.)

WARRANTY: This product carries a limited lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.

 Bodypoint®

 EC REP

 CE