

Sealed for safety

Cardinal Health™ ChemoPlus™ IVA Security Seals promote safety and security of IV admixtures

These tamper-evident seals are a solution to support your facility's USP <797> and <800> compliance practices. Available in a variety of sizes and colors for use with vials, IV bags and medication containers.



Sterile

- Protects point of entry

Dual warning system

- Ensures reconstituted medication is not further diluted or enhanced prior to administering to a patient
- Tamper-evident seal self destructs upon attempted removal
- Needle punctures are highly visible

Visual indicator

- Aids in the identification of contents, routes of administration, beyond-use dates or other alerts




Made in the U.S.

Cardinal Health™ ChemoPlus™ IVA Security Seals

Instructions for use

Cardinal Health™ ChemoPlus™ IVA Security Seals promote safety and security of IV admixtures

These tamper-evident seals are a solution to support your facility's USP <797> and <800> compliance practices. Available in a variety of sizes and colors for use with vials, IV bags and medication containers.

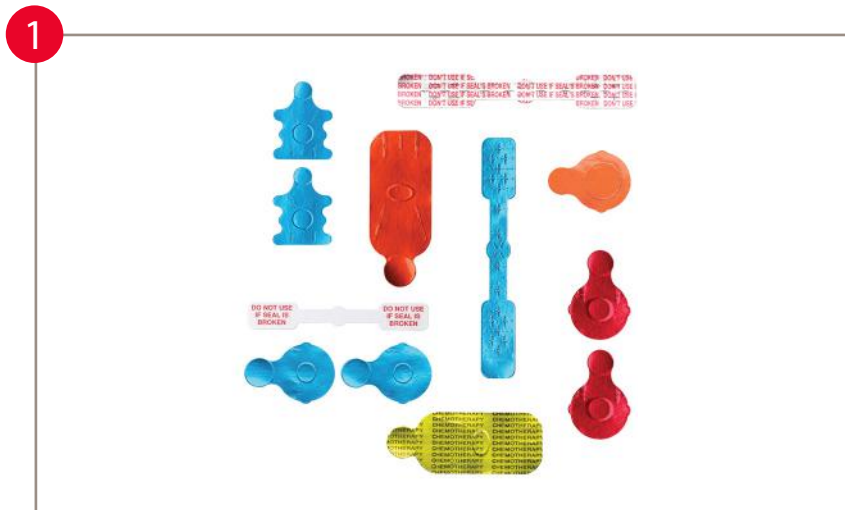

Select an appropriate ChemoPlus™ IVA Security Seal based on medication container type, size and color established by your facility's protocol. See full product offering on back page.



Prepare package for placement in compounding area following your facility's approved protocol.



Open clamshell package, position strip of seals in slot to use as dispenser and close, ensuring snaps are engaged.



Using aseptic technique, grab small pull tab and gently peel seal away from the backer strip.



Affix seal to medication container over target area. Press entire seal firmly to ensure seal is secure.




Educate your clinical staff on designations for selected colors and signs of tampering on seals.

Cardinal Health™ ChemoPlus™ IVA Security Seals

Ordering information

	Code	Description	Color	Seals/ carton
For vial tops				
	CP3005A-	9 mm	Silver	1,100
	CP3005R-	9 mm	Red	1,100
	CP3003A-	13 mm	Silver	1,100
	CP3003G-	13 mm	Green	1,100
	CP3003B-	13 mm	Blue	1,100
	CP3003R-	13 mm	Red	1,100
	CP3003Y	13 mm	Yellow	1,100
	CP3002Y-	20 mm	Yellow	1,000
	CP3002A-	20 mm	Silver	1,000
	CP3002G-	20 mm	Green	1,000
CP3002B-	20 mm	Blue	1,000	
CP3002O-	20 mm	Orange	1,000	
CP3002R-	20 mm	Red	1,000	
For bottle tops				
	CP3001A-	28 mm	Silver	1,000
	CP3001G-	28 mm	Green	1,000
	CP3001B-	28 mm	Blue	1,000
	CP3001Y-	28 mm	Yellow	1,000
	CP3001O-	28 mm	Orange	1,000
	CP3001R-	28 mm	Red	1,000
	CP3001T-	28 mm	Tan	1,000
	CP3009A-	36 mm	Silver	1,000
	CP3009G-	36 mm	Green	1,000
CP3009B-	36 mm	Blue	1,000	
CP3009R-	36 mm	Red	1,000	

	Code	Color	Seals/ carton
For Pfizer Large Bags and ADD-Vantage™ System			
	CP3004A	Silver	1,000
	CP3004G-	Green	1,000
	CP3004B-	Blue	1,000
	CP3004O	Orange	1,000
	CP3004R-	Red	1,000
For Baxter Viaflex™ Bags and Mini-Bag Plus Containers			
	CP3011A-	Silver	1,000
	CP3011G-	Green	1,000
	CP3011B-	Blue	1,000
	CP3011Y-	Yellow	1,000
	CP3011R-	Red	1,000
CP3111Y-	Yellow/black ink "Chemotherapy"	1,000	
For B. Braun Excel™ Bags and Small Bags			
	CP3013A-	Silver	1,000
	CP3013G-	Green	1,000
	CP3013B-	Blue	1,000
	CP3013R-	Red	1,000
For smaller syringes, 3.625 in. length			
	CP3030R-	Red	1,000
	CP3730A-	Silver/red ink "Don't use if seal's broken"	1,000
For larger syringes, 4.25 in. length			
	CP3020B-	Blue	1,000
	CP3020Y-	Yellow	1,000
	CP3020R-	Red	1,000
	CP3720A-	Silver/red ink "Don't use if seal's broken"	1,000