



Padded Reducer Ring Instructions

Model #4960, 4965

Thank you for choosing Columbia Medical

Your Columbia Medical Padded Reducer Ring has been built to the highest standards of quality to ensure years of superior service. Please take the time to read this informative instruction guide and pay particular attention to the safety instructions provided. Thank you again for choosing Columbia Medical.

Important Safety Instructions

When using your Padded Reducer Ring, basic safety precautions should always be followed to reduce the risk of injury to individuals.

1. Read and understand all instructions.
2. Determine, with a physician or therapist, the suitability of this reducer ring for the intended user.
3. Periodically check bolt that secures the abductor is tight at all times.

This reducer ring works well on most home toilet seats and is offered in two sizes:

- Model #4960: opening is 7" (W) x 8" (L)
- Model #4965: opening is 5" (W) x 6" (L)

These reducer rings feature non-slip seats and large removable abductor/splashguard.

If you would like to upgrade to a larger padded reducer ring, contact your Columbia Medical Customer Service Representative at (310) 454-6612.

Cleaning Your Padded Reducer Ring

Soapy water or mild cleansers and disinfectants may be used on the Padded Reducer Ring. Do not use harsh abrasive cleaners as these may damage the finish.
