

COMFORT FOOT SINGLE

WARNING: Make sure all screws are tightly clamped upon completion.

Clamp On

Parts Included:

1 - Clamp Assembly

Diagram 1A (steps 1-3)


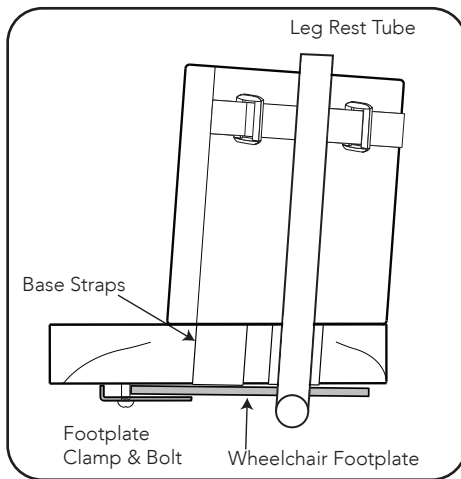


Diagram 2A (step 4)

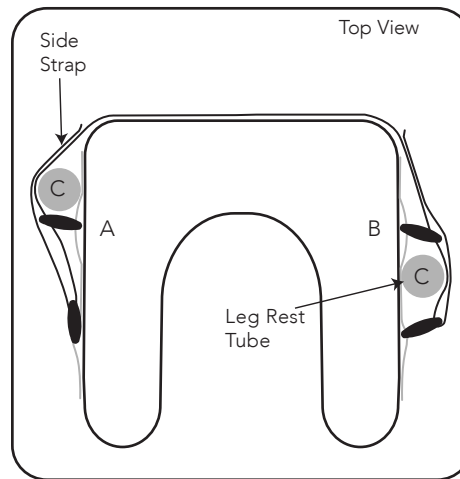

- 1** Loosen (do not remove) footplate clamp bolt using Phillips head screw driver.
- 2** Slide clamp over wheelchair footplate as seen in Diagram 1A.
- 3** Adjust Comfort Foot Single angle to desired position using base straps and tighten clamp bolt securely.
- 4** Release side strap and secure tightly around leg rest tubing (C). Note: Depending on the wheelchair, strap will have to be tightened in either configuration (A) or (B) as seen in Diagram 2A.

Bolt On

Parts Included:

2 - 1/4-20 x 1" or 1/4-20 x 1 1/4" Screws (dimensions vary depending on style of part)

2 - 1/4" x 1 1/4" OD Fender Washer

Diagram 1B (steps 3-4)


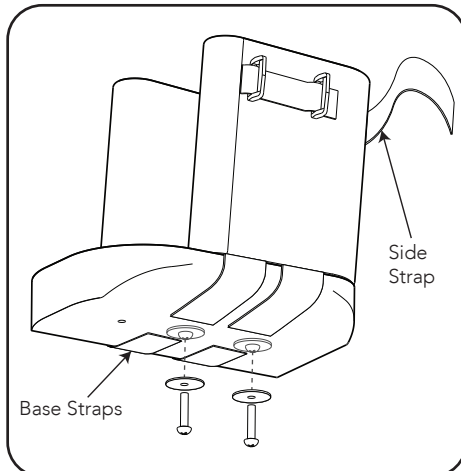


Diagram 2B (step 5)



- 1** Place the provided template on the wheelchair footplate in the desired position.
- 2** Drill two 5/16" diameter holes through the footplate as marked on the template.
- 3** Adjust Comfort Foot Single to desired angle using base straps.
- 4** Screw two 1/4-20 x 1" or 1/4-20 x 1 1/4" bolts with the provided washers from the bottom of the footplate into the Comfort Foot Single base. Note: Use 1" bolts for Comfort Foot Single with Gel.
- 5** Release side strap and secure tightly around leg rest tubing (C). Note: Depending on the wheelchair, strap will have to be tightened in either configuration (A) or (B) as seen in Diagram 2B.

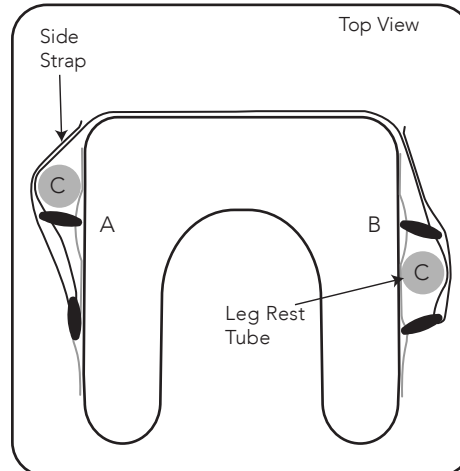

Comfort Foot Single[®]

BOLT-ON TEMPLATE

