

Three-in-One Commode

Silla inodoro tres en uno/
Chaise d'aisance trois-en-un



THREE-IN-ONE COMMUNE DIAGRAM

Silla inodoro tres en uno/Chaise d'aisance trois-en-un



WEIGHT CAPACITY:

350 lb/158 kg (BS31C shown above)

450 lb/204 kg (BSB31C)

Capacidad de peso / Capacité de poids

	English	Español	Français
1	Lid	Tapa	Couvercle
2	Backrest support	Soporte de respaldo	Dossier
3	Seat	Asiento	Siège
4	Frame	Estructura	Châssis
5	Leg tip	Puntas de la pata	Embouts
6	Commode pail with splash guard and lid (BS31C) Commode Pail only (BSB31C)	Cubo de silla inodoro con protector para salpicaduras y tapa (BS31C) Solo cubo de inodoro (BSB31C)	Seau de chaise d'aisance avec pare-éclaboussures et couvercle (BS31C) Seau de chaise seulement (BSB31C)
7	Height adjustable legs	Patas de altura regulable	Pattes à hauteur réglable
8	Back frame rail	Riel trasero de la estructura	Rail de châssis arrière
9	Armrest	Apoyabrazos	Accoudoirs

Congratulations on purchasing a quality ProBasics® Three-in-One Commode. Our commodes have a versatile design that can be used as a bedside commode, raised toilet seat or toilet safety frame. Please be sure to read these instructions and warnings thoroughly and follow the instructions of your healthcare provider to maximize the commode's benefits and your safety.

DO NOT ATTEMPT TO ADJUST OR OPERATE THIS COMMODE WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

ASSEMBLY INSTRUCTIONS

Remove the three-in-one commode components from all packaging including tape, cardboard and plastic. Check all parts for damage before use. If any parts are missing or damaged, **DO NOT** attempt to use the commode. If you do not understand the manual, contact your dealer for a complete explanation of the instructions, cautions and warnings. Otherwise injury may result.

NOTE: DO NOT use excessive force when installing any parts.

SETUP INSTRUCTIONS

1. Remove frame, seat, lid, pail, cover and splash shield from carton. Allow the seat and lid to reach room temperature if exposed to cold. This will help prevent the seat and lid clamps from cracking while being assembled onto commode back frame rail.
2. Align the toilet seat clamps onto the back-frame rail (both clamps should sit outside the pail brackets). Push down on seat with both hands until clamps snap firmly into place.
3. Repeat the same process for the lid, ensuring the lid clamps line up outside of the seat clamps.
4. Lift the seat and lid to rest against the backrest.
5. Place the commode pail between the pail brackets on the commode frame. Ensure the bucket lies flat on the frame. Remove the bucket lid before use.

ADJUST COMMODOE HEIGHT

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole.
2. Repeat for other three leg attachments.

CAUTION: Ensure that all four commode legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

TO USE COMMODE OVER TOILET

(AS A RAISED TOILET SEAT OR TOILET SAFETY FRAME)

1. Set commode to desired height (see "Adjust Commode Height" section).
2. Position splash guard in place onto pail supports.
3. Remove back support by simply unscrewing the wing nut assembly on either side.
4. Lift toilet bowl seat and place commode over toilet with splash shield positioned over the center of the toilet bowl.
5. Lower the commode seat onto the frame before use.
6. Clean seat and splash shield with disinfectant after each use.

TO USE AS A BEDSIDE COMMODE

1. Set commode to desired height (see "Adjust Commode Height" section).
2. Push commode pail onto pail supports. Ensure that pail is properly positioned below commode opening. Commode is now ready for use.
3. When user is finished using commode, place commode pail lid over commode pail to contain odors.
4. Remove pail and lid simultaneously to empty pail. Clean seat and pail with disinfectant after each use.

CAUTION: It is **NOT** recommended to remove the backrest support when using as a bedside commode. Serious injury may result.

MAINTENANCE

1. Use a mild soap and water solution, rinse thoroughly with clean water, then dry with a soft cloth. Ensure that commode is well rinsed and dried before using.
2. Never use organic solvents, abrasive cleaners or scouring pads on any part of commode.
3. Check fit and tightness of all nuts, bolts and knobs **WEEKLY** to ensure that the commode is stable and safe to use.



WARNINGS

DO NOT ATTEMPT TO ADJUST OR OPERATE THE COMMODE WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

- Maximum weight capacity: 350 lb/158 kg (BS31C), 450 lb/136kg (BSB31C).
- **DO NOT** use this device if it is not securely installed.
- This product should not be used without instruction from a healthcare professional.
- The toilet seat must be in down position before use. Failure to do so may result in injury.
- Ensure the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure the leg extensions are locked in position and an even height is achieved.
- **NEVER** stand on the commode seat or frame.
- All four leg extensions with rubber tips must touch the floor at all times.
- Users with limited physical strength should be supervised or assisted while using commode.
- Always use caution when floor is wet. Ensure armrest handles are dry before use.
- Use the armrest for assistance only. **DO NOT** use the armrests to support full body weight.
- Weight should be evenly distributed when using the commode. **DO NOT** lean or reach for objects while on the commode.
- Inspect all parts for shipping damage before assembly. If there is shipping damage — **DO NOT USE**. Contact dealer/carrier for further instructions.
- Users with limited physical capabilities should be supervised or assisted in commode areas even when using support rails.
- Consult a healthcare professional for advice and instructions specific to your needs before use.
- Compass Health Brands assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

USE CAUTION. FAILURE TO USE COMMON SENSE AND HEED THE ABOVE WARNINGS FURTHER INCREASES RISK OF SERIOUS INJURY. USE AT YOUR OWN RISK AND WITH APPROPRIATE AND SERIOUS ATTENTION TO SAFE OPERATION.