


Shower Chair

Silla de ducha/
Chaise de douche



SHOWER CHAIR DIAGRAM

Silla de ducha/Chaise de douche

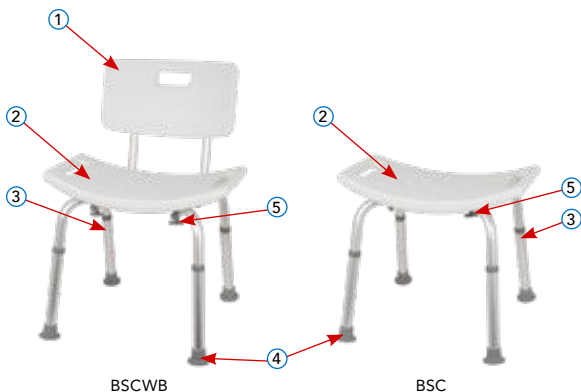

WEIGHT CAPACITY:
300 lb/136 kg (BSCWB, BSC)
Capacidad de peso / Capacité de poids

	English	Español	Français
1	Back support*	Soporte para la espalda	Dossier
2	Seat	Asiento	Siège
3	Height adjustable leg	Pata de altura regulable	Pattes à hauteur réglable
4	Leg tip	Puntas de la pata	Embouts
5	Thumb screw	Tornillo de apriete	Vis de serrage

*The back support is included with item #BSCWB.

*El soporte para la espalda está incluido con el producto n.º BSCWB.

*Le dossier est inclus avec l'article BSCWB.

Congratulations on purchasing a quality ProBasics® Shower Chair. Our shower chairs offer stability, comfort and easy assembly. Please be sure to read these instructions and warnings thoroughly and follow the instructions of your healthcare provider to maximize the shower chair's benefits and your safety.


⚠ DO NOT ATTEMPT TO ADJUST OR OPERATE THIS SHOWER CHAIR WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

ASSEMBLY INSTRUCTIONS

Remove the shower chair components from all packaging including tape, cardboard and plastic. Check all parts for damage before use. If any parts are missing or damaged, **DO NOT** attempt to use the shower chair. If you do not understand the manual, contact your dealer for a complete explanation of the instructions, cautions and warnings. Otherwise injury may result.

NOTE: DO NOT use excessive force when installing any parts.

SETUP INSTRUCTIONS

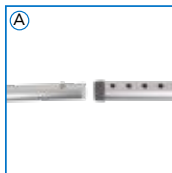


1. Remove seat frame, legs and back support (BSCWB only) from carton.
2. Turn seat upside down on a stable work surface. Insert leg into the leg frame by depressing push pins, sliding leg over receptacle until the push pin snaps into place. An audible click will be heard (Fig. A).

3. Repeat for the remaining 3 leg attachments.
4. Attach the backrest (BSCWB only) by depressing push pins and sliding backrest bars into frame until push pins snap into place.

HEIGHT ADJUSTMENT

1. Insert legs into the leg frames by depressing push pins, sliding legs over receptacle.
2. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired setting.
3. Repeat for other three leg attachments.
4. To ensure legs are locked into place, push down on seat or pull gently on each leg.

 **CAUTION:** Ensure that all four shower chair legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use. **DO NOT** sit on shower chair if legs are not evenly adjusted, locked and stable.

MAINTENANCE

1. Check fit and tightness of all thumb screws and push pins **WEEKLY** to ensure that the commode is stable and safe to use.
2. **NEVER** use organic solvents, abrasive cleaners or scouring pads on any part of shower chair.
3. Use a mild soap and water solution, rinse with clean water, then dry with a soft cloth. Ensure that the shower chair is well rinsed before using.

WARNINGS

DO NOT ATTEMPT TO ADJUST OR OPERATE THE SHOWER CHAIR WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

- Maximum weight capacity: 300 lb/136 kg (BSCWB & BSC).
- **DO NOT** use this device if it is not securely installed.
- This product should **NOT** be used without instruction from a healthcare professional.
- The seat and legs **MUST** be securely attached to the frame prior to use. Failure to do so may result in injury or damage to the shower chair.
- Ensure the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure the leg extensions are locked in position and an even height is achieved.
- All four leg extensions with rubber tips **MUST** touch the floor at all times.
- Ensure **ALL** thumb screws are tightened and threading is intact on a weekly basis.
- Check all the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
- Weight should be evenly distributed when using the shower chair. **DO NOT** lean or reach for objects while seated on the shower chair.
- **NEVER** stand on the shower chair seat.
- It is recommended that the user ease down onto the seat. **DO NOT** sit on the seat with direct force.

- **ALWAYS** use caution when surfaces are wet.
- Inspect all parts for shipping damage before assembly. If there is shipping damage — **DO NOT USE**. Contact dealer/carrier for further instructions.
- Users with limited physical capabilities should be supervised or assisted in shower or tub areas even when using support rails.
- Accessories designed by other manufacturers have not been tested by Compass Health Brands and are not recommended for use with this product.
- Consult a healthcare professional for advice and instructions specific to your needs before use.
- Compass Health Brands assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- Shower Chair is **NOT** designed to be used as a transfer device, pool chair or long-term seating system. Shower chair is not designed to be used outside of bathroom

USE CAUTION. FAILURE TO USE COMMON SENSE AND HEED THE ABOVE WARNINGS FURTHER INCREASES RISK OF SERIOUS INJURY. USE AT YOUR OWN RISK AND WITH APPROPRIATE AND SERIOUS ATTENTION TO SAFE OPERATION.