


K1 WHEELCHAIR

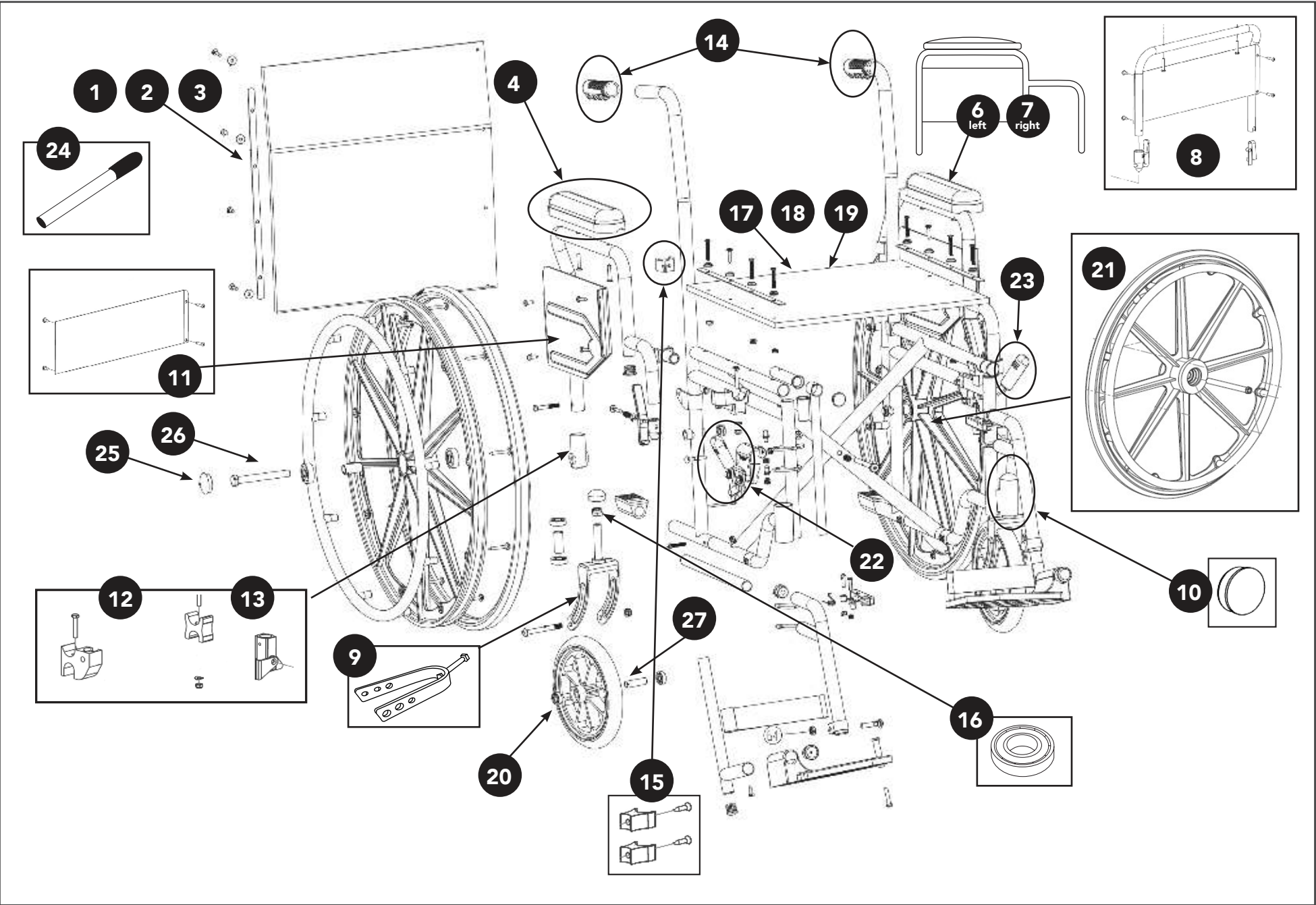

K1 WHEELCHAIR

Fig.	Description	MH Part No.
K1 REPLACEMENT PARTS		
1	Probasics K1 Wheelchair 16" Back Upholstery w Hardware	X1WC1IM01
2	Probasics K1 Wheelchair 18" Back Upholstery w Hardware	X1WC1IM02
3	Probasics K1 Wheelchair 20" Back Upholstery w Hardware	X1WC1IM03
4	Probasics K1 Wheelchair Desk Length Armrest Pads, Pair w/Hardware (not full arm)	X1WC1IM04
5	Probasics K1 Wheelchair Full Length Armrest Pads, Pair w/Hardware (not full arm), <i>not pictured</i>	X1WC1IM05
6	Probasics K1 Wheelchair Replacement Armrest Assembly, Desk, Left	X1WC1IM06
7	Probasics K1 Wheelchair Replacement Armrest Assembly, Desk, Right	X1WC1IM07
8	Probasics K1 Wheelchair Full Length Armrest Assembly, Pair	X1WC1IM08
9	Probasics K1 Wheelchair Front Fork Assembly	X1WC1IM09
10	Probasics K1 Wheelchair Stem Dust Cap for Front Fork	X1WC1IM10
11	Probasics K1 Wheelchair Skirt Guard w/Hardware, Pair	X1WC1IM11
12	Probasics K1 Wheelchair Right Flip-Back Armrest Detach. Guides Front&Rear w/Hardware	X1WC1IM12
13	Probasics K1 Wheelchair Left Flip-Back Armrest Detach. Guides Front&Rear w/Hardware	X1WC1IM13
14	Probasics K1 Wheelchair Hand Grips, Pair	X1WC1IM14
15	Probasics K1 Wheelchair Seat Rail Guide, Pair	X1WC1IM15
16	Probasics K1 Wheelchair Fork Stem Bearings	X1WC1IM16
K1/K2 COMPATIBLE PARTS		
17	Probasics K1/K2 Wheelchair 16" Seat Upholstery w/Hardware	X1WCIM01
18	Probasics K1/K2 Wheelchair 18" Seat Upholstery w/Hardware	X1WCIM02
19	Probasics K1/K2 Wheelchair 20" Seat Upholstery w/Hardware	X1WCIM03
20	Probasics K1/K2/K4T Front Caster Wheels w/Bearings (Bearing Size - 8mm x 22mm x 9mm)	X1WCIM04
21	Probasics K1/K2 Rear Wheel w/Bearings (Bearing Size - 35mm x 11.1mm x 11mm)	X1WCIM05
22	Probasics K1/K2 Wheelchair Wheel Lock, Right	X1WCIM06
23	Probasics K1/K2 Wheelchair Wheel Lock, Left	X1WCIM07
24	Probasics K1/K2/R Wheelchair Wheel Lock Extension	X1WCIM09
25	Probasics K1/K2 K4T/R Dust Cap for Rear Wheel	X1WCIM10
26	Probasics K1/K2/R Wheelchair Rear Wheel Axle Bolt w/Nut (110MM Length 7/16" Nut)	X1WCIM11
27	Probasics K1/K2/K4T/R Caster Axle Bolt w/Nut	X1WCIM12
ACCESSORIES		
	Elevating Legrest	ELR-C
	Swing away legs	RSA-C / X1WCIM08
	Anti-tippers	X1WCIMANTI
	Seat Belt	PB3065