



Now You're Going Places

CRUISER

Replace One-Piece Push Handle

Tools required:

- Phillips screwdriver
- 3/8" wrench

1. Remove screw attaching pivot joint to frame with Philips screwdriver and 3/8" wrench (both sides). **Fig. 1**
2. Pull Joint back and remove after unscrewing screw from both sides of handle. **Fig. 2**
3. Replace the new one piece handle by attaching to frame. **Fig. 3**
4. Screw both sides of handle to frame at pivot joint with Philips screwdriver and 3/8" wrench. **Fig. 4**
5. Final product. **Fig. 5**



Fig. 1



Fig. 2



Fig. 3



Fig. 4



Fig. 5