

# HOW TO RESET A BELLAVITA HAND CONTROL

*Instructions and test report for Handcontrol of  
ENDRES AKKULIFT® - Bathlift (all Models)*





## RESET GUIDE FOR BELLAVITA HAND CONTROL

*THE HAND CONTROL MUST BE DISCONNECTED FROM THE BATHLIFT AND PERFORMED TWICE TO FULLY RESET*

1. Step: Press the Up & Down button at the same time and keep them pressed

2. Step: Keeping the Up & Down pressed then press the ON button for a moment and release, the test mode starts.

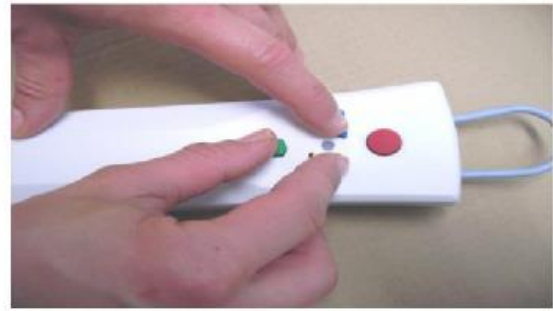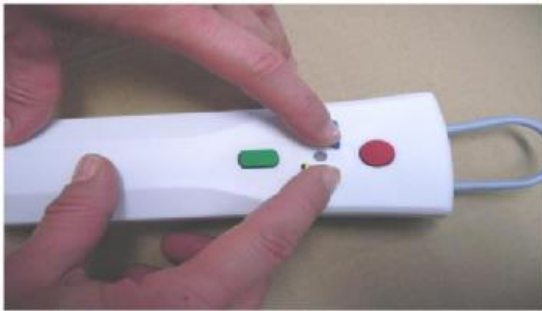

### Test / Reset Mode:

The LED initially flashes green, then to amber, and then blinks amber. Press the yellow UP button and you will hear a quiet click of the relay in the hand control (the UP movement is possible and the hand control is working), press the blue DOWN button (no click is to be heard).

Switch from Test Mode and reset the hand control by pressing the red EMERGENCY button and the hand control can be used again normally again (SEE ABOVE).