

drive

3 panel privacy screen

item # 13508

drive
MEDICAL DESIGN & MANUFACTURING

assembly instructions

1. Remove screen and caster box from carton.
2. Remove nuts and bolts and casters from box.
3. Place screen on it's side.
4. Slide a plastic cap over the leg.
5. Unscrew nuts from bolts and position one of the washers on the bolt.
6. Align one of the casters with the holes in the leg of the screen.
7. Insert bolt through the holes of the caster and screen leg. Be sure to position a washer in between the bolt head and the caster leg.
8. Place another caster leg over the end of the bolt that is protruding form the hole.
9. Place a washer over the end of the bolt.
10. Place a nut over the end of the bolt and use the provided allen wrench to tighten the bolt.
11. Repeat for other screen leg.
12. Make sure all bolts are firmly tightened.
13. Slide plastic cap down to cover top of caster legs.

