


# assembly instructions

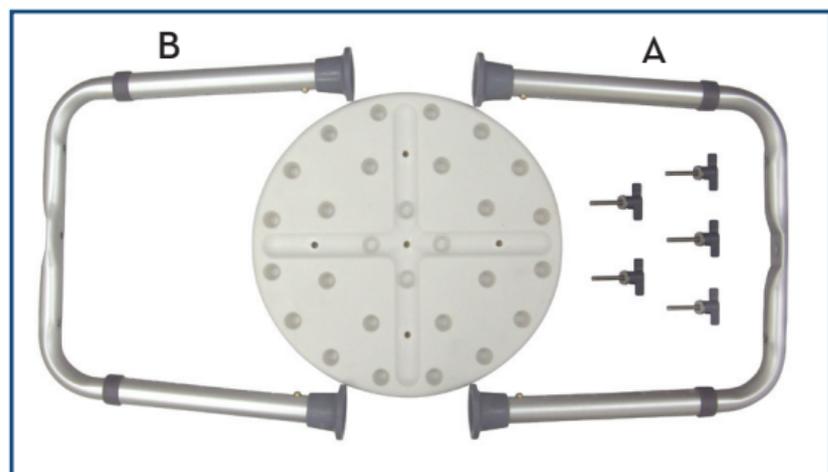

1. Remove contents from box.
2. Place seat upside down on a flat surface so holes are showing.
3. Align leg section A (the leg section that has the concave area on the underside) with the holes in seat. Place and tighten the two handscrews in the outer holes. Be sure to put on a metal washer and a plastic washer (in order). Do not affix the middle handscrew.
4. Align leg section B (the leg section that has the concave area on the top side) with the holes in the seat. Place and tighten the two outer holes using the handscrews and washers.
5. Place the middle handscrew and tighten through the two legs.

**NOTE:** Periodically check the handscrews to make sure they remain tight as loosening may occur.

