

## lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



folding cane  
sling seat

drive  
MEDICAL DESIGN & MANUFACTURING

# cane sling/seat

The Drive Cane Sling/Seat is a versatile and convenient seating device, when open and a strong quad cane, when closed. This device is intended for individuals who need to rest when walking a long distance. The four legs with vinyl tips serves as a better walking aid than most 3-legged canes. It is equipped with a durable nylon sling seat that wraps around and secures with a hook and loop closure when the seat is not in use. Individuals can comfortably hold onto this device on the padded foam grip handles.

Height (when folded): 34"

Height (seat to floor): 22"

Weight: 2.4 lbs

Seat Dimensions: 18" x 8"

## features

- No assembly required
- Unfolds easily in seconds
- Durable bronze aluminum construction
- Cushioned foam grip handles
- Nylon sling seat
- Can be used as a quad cane, when folded

**WEIGHT CAPACITY: 250 LBS.**

## proper sitting procedure

