

installation instructions


Weight Capacity: 20 lbs

1. Pick up head halter by two metal “D” rings, holding both with one hand, to make head halter hang as shown.
2. Place head through the middle with chin resting in chin cup and straps hanging from back of head “straps on outside”.
3. Fasten Velcro-type straps onto Chin Piece.
4. Adjust for correct size by moving overhead strap up or down at “D” rings.

NOTE: Adjust overhead strap as necessary.

Item # 13004
Shown

“D” Rings

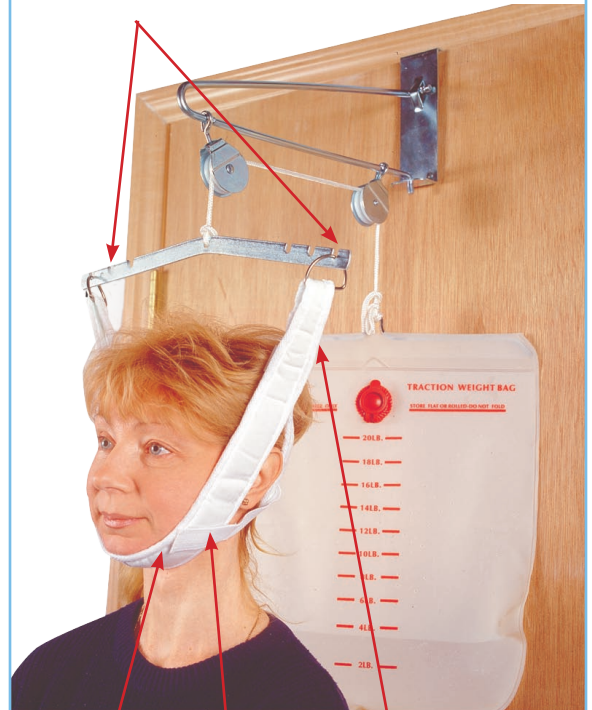

Chin Cup
Front of Halter

Velcro-type
Straps

Back Part of
Halter with
Straps on Outside