

assembly instructions

1. Remove contents from carton.
2. Lay transfer bench upside down on a flat surface.
3. Attach leg with the large suction cup by depressing the brass push pin and inserting into the receptacle. Repeat for second leg (Figure 1).
Note: These legs are inserted into the same side that the arm support will be.
4. Attach leg with small rubber tip by depressing brass push pin and inserting into the receptacle. Repeat for second leg (Figure 1).



figure 1

5. To assemble backrest, insert pole into backrest support by depressing push pin and sliding into receptacle. Repeat for second pole (Figure 2).

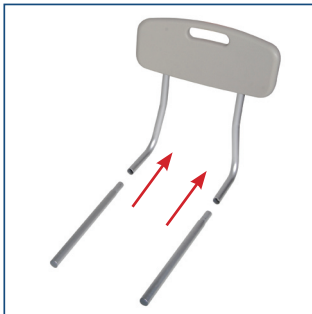


figure 2

6. Insert backrest into base by depressing push pins and sliding supports through both receptacles.
Note: backrest must go through both sets of receptacles to become secure (Figure 3).

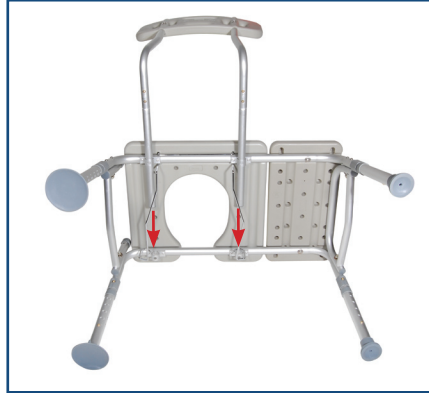


figure 3

7. Turn transfer bench right-side-up. Attach armrest by depressing brass push pins and sliding into the arm receptacle on the side of the bench. Make sure the arm arcs outward, not inward (Figure 4).



figure 4

8. Slide commode bucket into hanging metal bars and affix hooks over back support bars to secure in place (Figure 5).



figure 5

9. Place legs with suction cups into bath tub.
Note: Pull Tab on the suction cup to release from the tub.
Adjust legs to the need of the patient.
The transfer bench/commode is best used when the bench is level.

