


parts list

- A. Armrest/Handle
- B. Side Frame
- C. Screws
- D. U Shaped Support Frame
- E. Thumb Screws
- F. Basket

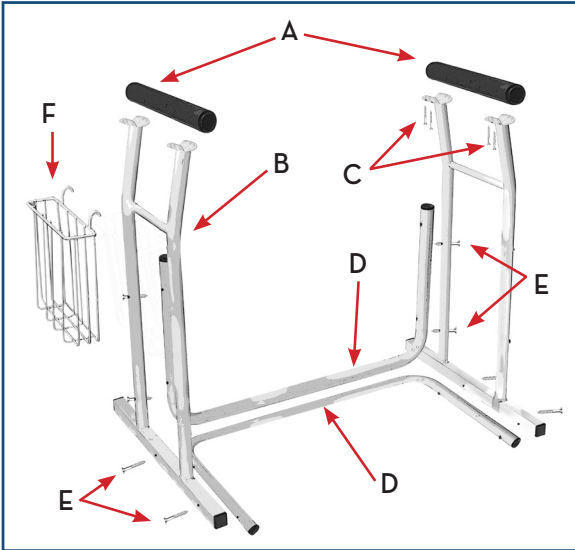

assembly instructions

WEIGHT LIMIT 300 LBS.

1. Remove contents from carton.
2. Attach armrest/handle (A) to side frame (B) using 2 screws (C). Repeat for second side frame.
3. Stand the side frame (B) on its base. Attach one of the U shaped support frames in the horizontal position (D) to bottom of base using 2 thumb screws (E).
4. Attach the second U shaped support frame (D) to the side frame in vertical position using 2 more thumb screws (E).
5. Repeat steps 3 and 4 for second side frame.
6. Hang magazine basket (F) on desired side of side frame.

Note: Check thumb screws periodically for tightness. Tighten as necessary.

