

The Drive Bath Step is designed to make getting in and out of the bath or tub easier. The Drive Bath Step features a large flat slip proof stepping surface with built-in non-slip pads. Two or more steps can be combined to raise the height of the steps by following the instructions below.

instructions

To combine bath steps, follow the instructions below.

1. Place bath step on flat surface.
2. Using a coin, remove each of the 4 white circles located on the top of the bath step. Store these for future use.



3. Place second bath step on top of first, lining up prongs on underside of step with receptacle holes. Press down firmly to secure in place.

