

SAFETY SUMMARY

To ensure the safe operation of DRIVE CANES, these instructions **MUST** be followed.

GENERAL WARNINGS

DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment-otherwise, injury may occur.

Check rubber tip for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tip immediately. Drive Canes can provide ambulatory assistance for an individual weighing up to 300 pounds. The canes are designed to provide support, increased stability and assistance while walking. The canes are NOT designed to support the total weight of individuals.

Ensure that the snap button fully protrudes through adjustment hole of cane. This ensures that the cane is securely locked in position. BEFORE using the cane, ensure that it is securely locked in place or a fall could result, causing bodily injury and/or damage to the cane.

ADJUSTING FOR RIGHT/LEFT HAND USE

1. Loosen lock knob
2. Depress the snap button and turn cane handle 180 degrees until snap button locks into same height adjustment hole on other side of cane.
3. When cane handle is firmly positioned at desired height and snap button protrudes through height adjustment hole on cane handle, turn lock knob clockwise to secure.

PROPER CANE HEIGHT

For a more comfortable fit, ensure shoe height is the same as shoe style worn most frequently when using cane.

1. Stand as upright as possible on a firm, flat surface and let your arm hang down naturally by your side.
2. Adjust cane so top portion of handle touches inside of wrist where it meets palm.




HEIGHT ADJUSTMENT

1. Turn the anti-rattle collar **COUNTERCLOCKWISE** to loosen.
2. Press **IN** snap button.
3. Slide extension up or down until snap button protrudes through an adjustment hole on handle end of cane.

NOTE: An audible “CLICK” will be heard when snap button securely protrudes through the adjustment hole.

4. Repeat **STEPS 2-3** until cane is at height determined **PROPER CANE HEIGHT** on this instruction sheet.
5. Tighten lock knob by turning **CLOCKWISE**.

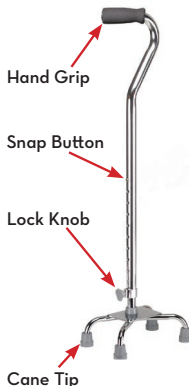

MAINTENANCE & CARE

1. Ensure snap button releases and re-engages properly.
2. Inspect cane tips for wear and damage.
3. Replace any broken, damaged or worn items immediately.

WEIGHT CAPACITY: 300 LBS.

ALWAYS CONSULT YOUR PHYSICIAN OR THERAPIST TO DETERMINE PROPER ADJUSTMENTS AND USAGE