

installation instructions

1. Remove product from carton and read directions carefully before installing raised toilet seat.
2. Remove toilet seat and lid from toilet.
3. Turn both locking clamps counter clockwise until they can no longer turn. This will open the locking devices.
4. Place raised toilet seat on bowl and position in a secure fashion. Be sure the two locking brackets are on both sides of the bowl wall and that the seat is flush to the bowl.
5. Turn the locking brackets clockwise until tight. Hand tighten only. Do not use tools as threads on adjustment bracket may strip.
6. Check for secure fit routinely before using product. User's weight should be centered over toilet seat.

removal instructions

1. Rotate knobs counter clockwise until seat is loose.
2. Lift front of raised toilet and pull forward to move.

cleaning instructions

1. Use a non-abrasive detergent or household cleaner with warm water.
2. Rinse with warm water and wipe dry.
3. **DO NOT USE AN ABRASIVE CLEANER OR CLOTH** with this product, as damage may occur.

CAUTION

1. Always be sure that the seat is correctly and securely locked in place before using.
2. Be sure that adjustment knobs on sides of seat are tight.
3. This product is designed for use with most common types of commodes. It will not fit all commode types; be sure that the unit is correctly installed, fits properly and is stable before use.