

lifetime limited warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

Warning: The items and packaging in this shipment contain chemicals known to the state of California to cause cancer, birth defects or other reproductive harm.

*This product meets or exceeds all federal safety and environmental regulations.



competitive edge rollator

item #1239RD, 1239BL



assembly instructions

Do not attempt to adjust or operate Rollator without carefully reading all instructions in this manual.

1. First remove all contents from carton: backrest, handle with tube, and Rollator frame.
2. Open frame and slide padded backrest into frame brackets making sure snap bottoms protrude through holes.
3. Loosen tightening knob on handle receptacle and insert handle tube into frame. Set handle to desired height. Tighten knob by turning clockwise until handle is securely in place. Make sure both handles are set to the same height before use.



safety precautions

1. Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
2. Be sure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
3. When wheels are locked (to stop Rollator apply weight downward on handlebars.), lock bar on rear wheels should prevent wheels from turning. If the wheels turn while brakes are locked, please take unit back to dealer for adjustment. When locks are released, the wheels should turn freely.
4. Always lock before sitting and do not use seat when unit is on an incline or uneven ground.
5. Replace tires when significantly worn or damaged.

Important Safety Notice

Do not have anyone push you while you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device.

to stop rollator

To stop Rollator apply weight downward on handlebars. This will engage tips and bring the Rollator to a stop.