

Do not attempt to adjust or operate Rollator without carefully reading all instructions in this manual.



assembly instructions

1. Remove all contents from carton.
2. Open frame and place up-side-down on the floor. Attach legs without brakes to the front of the walker by depressing push pin and sliding leg into the receptacle. Secure in place using the tightening knob, threading the knob in from the outside. Repeat for second leg, making sure both legs are set to the same height.
3. Attach back legs (with brakes) using the same process, making sure the wheels are on the outside of the rollator and are at the same height as the front wheels. Affix brake cable to the black portion of the leg using the “S” shaped clamp provided.
4. Insert handle tube into frame and set to desired height. Secure in place using tightening knob, threading the knob in from the outside. Repeat for second handle, making sure both are set to the same height.
5. Attach padded backrest by depressing the black thumb buttons and sliding into the black bracket. Note that back rest can be folded up when not in use.


safety precautions

1. Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
2. Be sure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
3. Test the loop locks before operating the Rollator. When wheels are locked (see illustrations for locking and releasing locks below), lock bar on rear wheels should prevent wheels from turning. If the wheels turn while brakes are locked, please take unit back to dealer for adjustment. When locks are released, the wheels should turn freely.
4. Always lock loop locks before sitting and do not use seat when unit is on an incline or uneven ground.
5. Replace tires when significantly worn or damaged.
6. Do not have anyone push you while you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device.

loop lock operation

Always engage and disengage wheel locks on both sides simultaneously.

1. To Slow Down or Come to a Temporary Stop – Pull up slowly on the loop locks with four fingers until unit slows down or comes to a temporary stop.
2. To Temporarily Lock Wheels in Position – Press down on both loop locks with palms until you hear a “click” that will tell you wheels are locked.
3. To Release Wheel Locks – Push up on both loop locks with palms of hand to original position.

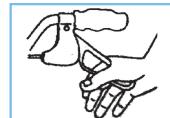

height adjustment

The height of the rollator can be adjusted in two ways to accommodate many different users.

handle height adjustment

1. Remove tightening knob from handle.
2. Set handle to desired height.
3. Replace tightening knob into handle, making sure to insert knob from the outside.
4. Repeat for second handle, making sure both handles are set to the same height setting.

leg height adjustment

1. Remove tightening knob from leg.
2. Depress push pin and slide leg to desired height.
3. Replace tightening knob into leg, making sure to insert knob from the outside.
4. Repeat for second, third and fourth legs, making sure all are set to the same height setting.

drive