

EasyStand Zing

MPS & Supine size1



Owner's Manual

SYMBOL LEGEND



CAUTION/ATTENTION



PINCHPOINT

SN

SERIAL NUMBER



Please be environmentally responsible and recycle this product through your recycling facility at its end of life.



ENVIRONMENTAL
CONDITIONS



DATE OF MANUFACTURE



SEE INSTRUCTIONS FOR USE

R_xonly

Table of Contents

Introduction	1
Safety Precautions	2
Assembly	3
Adjustments & Usage	9
Parts Detail	17
Maintenance	19
Limited Warranty	20

Date Purchased _____ Serial # _____

Introduction

Congratulations on your purchase of the EasyStand Zing standing frame. We at Altimate Medical have designed the EasyStand Zing with your needs in mind.

EasyStand Description

The EasyStand line of products is intended for individuals who are able to transfer with assistance from a caregiver and/or lifting device. It is a useful tool for rehabilitation, home, school, and work use. EasyStand standing frames support the user from various positions to the standing position. Various support and positioning options may be added to facilitate individual needs.

Indications for Use

The EasyStand Zing products have a flat surface that can be adjusted from the prone or supine position to increase tolerance to an upright standing position. The Zing Size 1 is indicated for most individuals weighing up to 70 lbs.

This manual is an integral part of your standing system and should be read completely and thoroughly by anyone that will be operating the unit. This will ensure the EasyStand Zing is assembled properly and used in the most beneficial manner.

The EasyStand Zing must be used with a qualified adult attendant present. Should any problem arise that you cannot solve by reading this manual, please contact your local supplier

It is necessary to inspect your EasyStand Zing at least weekly to ensure it is in safe operating condition. EasyStand adjustments that are listed in the product owner's manual can be made by the operator by following the instructions for use. After an adjustment is made, inspect the EasyStand to ensure all hardware and adjustable components are properly secured per the instructions for use. Replacement of worn parts, major adjustments, or any other important corrections should be handled by an authorized supplier or service center. Only Altimate Medical, Inc., approved replacement parts should be used to ensure safety and performance.

Do not use the EasyStand Zing without consulting your physician. Failure to consult a qualified physician prior to using the EasyStand Zing could lead to severe medical problems. We recommend a good standing program be established and supervised by a qualified physician or occupational/physical therapist. only

Join the Standing Community!

Our goal is simple - to empower people with disabilities to stand up for their health. Connect, share, and learn from other people with disabilities, special needs parents, clinical professionals, and the experts in standing on one of our social networking communities.


Safety Precautions

- Please read and understand all safety precautions and user instructions prior to use.
 - If a user does not meet the specifications below and has not been properly fitted by a qualified therapist or physician, Altimate Medical, Inc., does not recommend using the EasyStand Zing.
Zing Size 1 - is designed to accommodate most individuals up to 44" (112 cm) and up to 70 lbs. (32 kg).
 - The EasyStand Zing should not be used without first consulting a qualified physician. Failure to consult a qualified physician prior to using the EasyStand Zing could lead to severe medical problems. We recommend a good standing program be established and supervised by a qualified, registered physical/occupational therapist.
 - The EasyStand Zing should always be used with a qualified adult attendant present. Consult with your therapist or physician for proper foot placement and angle. Never leave a child unattended in the Zing.
 - Do not make foot plate adjustments to the Zing while a child is standing.
 - It is necessary to inspect your EasyStand Zing at least weekly to ensure it is in safe operating condition. Pay particular attention for loose hardware. Replacement of worn parts, major adjustments, or any other important corrections should be handled by an authorized dealer or service center. Only Altimate Medical, Inc. approved replacement parts should be used to ensure safety and performance.
 - If your EasyStand is being used in the home environment, please inspect your unit prior to each use to ensure there is no damage or unexpected wear to the EasyStand that may have been caused unintentionally by pets, pests or children.
 - Ensure the necessary support and alignment options are in position prior to placing the user into the standing position.
 - Storage/Transport Conditions: Temperature: 14°F -122°F (-10°C - 50°C), Relative humidity: 20% - 90% at 86°F (30°C), Atmospheric Pressure: 700 to 1060hPa.
 - If the unit was exposed to temperatures of less than 50°F (23.5°C), allow unit to warm to room temperature before use.
 - The expected service life of an EasyStand is considered 5 years under normal use conditions. Note: This may vary based on frequency of use.
 - Standing equipment is designed for indoor use on level surfaces only. Keep casters locked while in use.
 - It is not recommended to transport the unit with a user in the standing position.
 - Extreme movements laterally or front to back may cause the unit to tip.
 - It is recommended the equipment be stored at normal room temperature and humidity levels.
 - Altimate Medical products are specifically designed to be used with Altimate Medical accessories and options. Unless otherwise noted, accessories and options from other manufacturers have not been tested by Altimate Medical and are not recommended for use with Altimate Medical products.
 - Product Modifications made without express written consent (including, but not limited to, modification through the use of unauthorized parts or attachments) are not recommended and will void the product warranty.
-  **Caution:** This equipment is designed to accommodate a range of users; therefore the unit has various adjustment slots and holes. Avoid placing fingers or other body parts in any of these areas.
-  **Caution:** While in use this equipment has various moving parts, familiarize yourself with the unit and avoid placing hands or fingers near any of these areas.
-  **Caution:** Advise bystanders to keep clear of the unit while in use.

Assembly

1. Disregard all clear plastic bags and wrapping on the product. Lay out the parts as shown:

- A. Base Wheel Assembly
- B. Mast
- C. Foot Plates
- D. Hip Supports
- E. Calf Pads
- F. Knee Pads
- G. Lateral Supports
- H. Tray

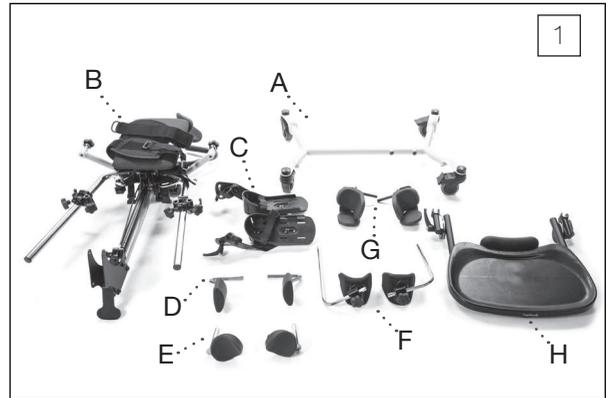


2. Remove the bag with the 4 bolts and washers from the base wheel assembly and set aside.



3. If the gas spring lockout was ordered, make sure the gas spring lockout located on the mast is in the unlocked position as shown.



4A. Place the mast onto the floor as shown. Depress the foot pedal.

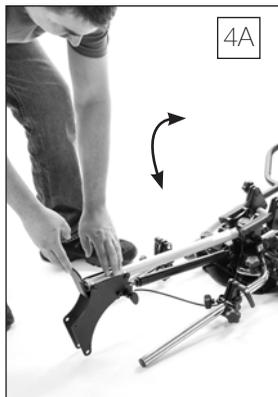

4B. Allow the mast post to come up to a vertical position.



5. Flip the base wheel assembly over and place onto the mast as shown.



6. Start all 4 bolts and washers you removed in step 2. Do not tighten at this time.



7. Once all 4 bolts have been started, tighten all 4 bolts securely. Place unit onto its wheels.



8. If the calf pad with knee strap was ordered, place the assembly onto the legs as shown.




9. Once the calf pad with knee strap is in position, tighten knobs securely.




10. If the independent adjustable calf pads were ordered, loosen the ratchet handles on the clamps and position as shown. Insert the calf pads into the clamps on the multi-adjustable knee pads.



11. Insert multi-adjustable knee pads into the bracket on the outer side of the legs. Tighten securely.

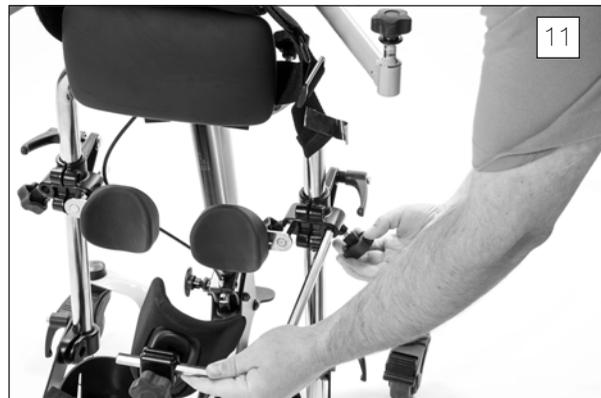

12. If the platform foot plate was ordered, place the assembly onto the legs as shown. Tighten securely.

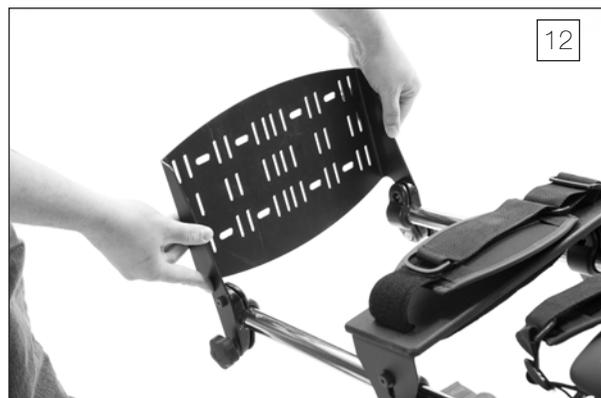


13. If the multi-adjustable foot plates were ordered, place each foot plate onto the legs. Tighten knobs securely.



14. If the hip supports were ordered, insert hip supports into the bracket behind the pelvic pad. Tighten securely.



15. If the lateral supports were ordered, insert lateral supports into the bracket behind the upper body support pad. Tighten securely.

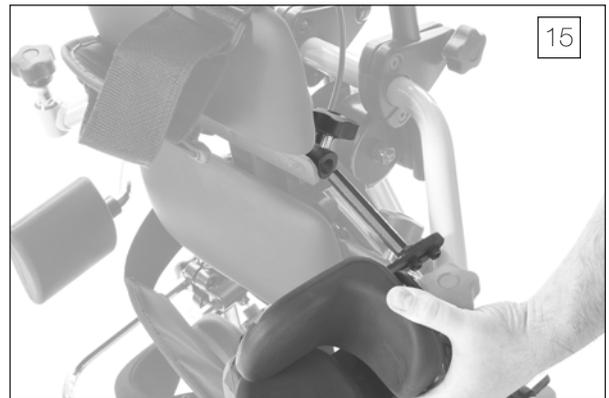

16A. If the black molded tray was ordered, insert the two tray arms into the tray bracket as shown.



16B. Flip the red plunger back into the locked position and tighten knob securely.



17A. If the multi-adjustable tray was ordered, there are 2 knobs in the parts bag. Screw each knob into the tray bracket on the unit.



17B. Place the tray bracket assembly into the base. Tighten securely.



18. Remove head support and place the stem into the clamp as shown. Tighten securely.




Adjustments & Usage

Trays

Black Molded/Multi-Adjustable Tray Height

19. To adjust tray height, loosen knobs on the back of unit and set to desired position. Tighten securely.

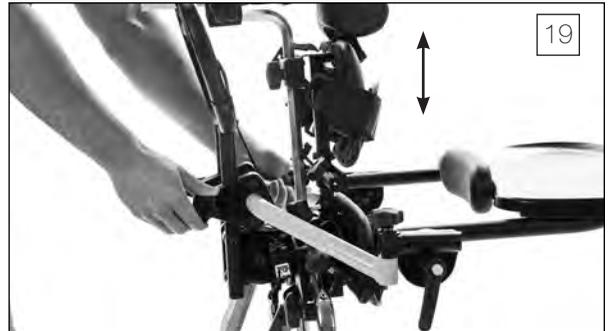


Black Molded Tray Depth

20. To adjust the tray, loosen the knob until resistance is felt, then flip the red lever in the center of the knob fully open.


⚠ DO NOT continue to turn the knob counterclockwise. Forcing the knob past its stop will damage the mechanism.

The tray may now be removed completely, or adjusted to the desired position using the click detent positions as a guide.

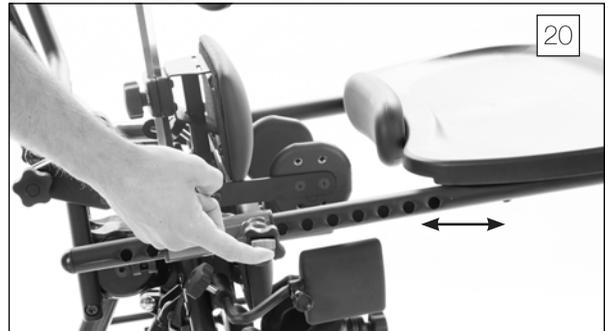

Multi-Adjustable Tray Depth

21. Loosen knob under the tray, pull or push into desired position. Tighten securely.

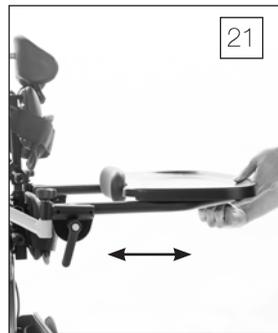

Multi-Adjustable Tray Angle

22. To adjust tray angle, loosen ratchet handles on both sides of the tray, set to desired position. Tighten securely.




Multi-Adjustable Tray Swing-Away

23. Loosen knob on one side only until you are able to swing tray away.



24. Swing tray away to allow for transfers.

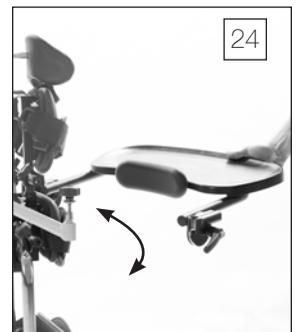


Hip Supports

25. The hip support width can be adjusted by loosening the knobs behind the pelvic pad. Hip supports come with both thumb knobs and set screws for fore/aft positioning. Tighten securely.




Lateral Supports

26. The lateral support width can be adjusted by loosening the knobs on the back of the unit. Always tighten securely. The depth can be adjusted by loosening the two bolts, once the desired position is reached. Tighten securely.

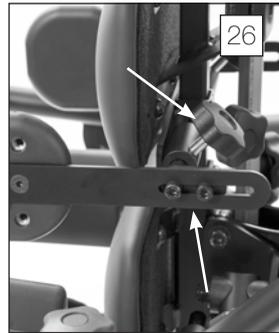

27. The lateral supports have a number of attachment holes on the back of the pad which will allow for additional positions vertically and horizontally. The pads may also be bolted at a high, low, or neutral position in order to effect greater pad range.

28. To adjust the lateral support height, loosen knob, adjust to desired position. Tighten securely.

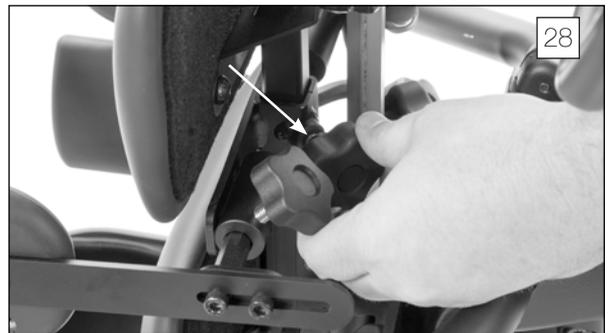


Calf Pad with Knee Strap

29. To adjust the calf pad with knee strap, loosen knobs on each side. Position fore/aft and up/down to the desired setting. Tighten securely.




Multi-Adjustable Knee Pads

30. To adjust the multi-adjustable knee pad height, loosen the ratchet handle. Adjust to desired position. Tighten securely.

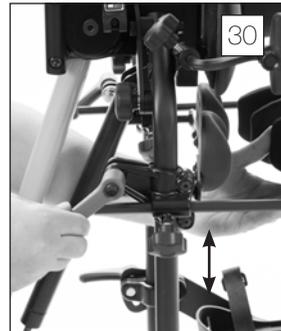

31. To adjust the multi-adjustable knee pad depth, loosen the knob. Adjust to desired position. Tighten securely.

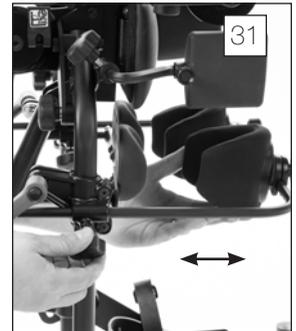

32. To adjust the multi-adjustable knee pad position, loosen knob and rotate to desired position. Tighten securely.




Independent Adjustable Calf Pads

33. To adjust the independent adjustable calf pad depth, loosen the knob. Adjust to desired position. Tighten securely.



34. Remove bolts and position as desired to adjust calf pad height. Tighten securely.

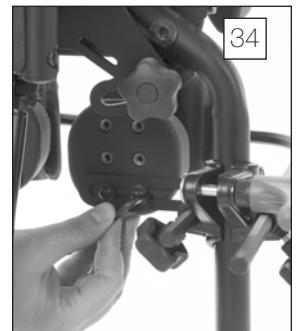

Head Support


35. To adjust the head support height, loosen the knob on the back, move to desired position. Tighten securely.

36. To adjust the head support depth, loosen the knob on the back of the headrest and slide into desired position. Tighten securely.

To adjust the head support angle, loosen the ratchet handle and place into desired position. Tighten securely.


Upper Body Support Pad

37. To adjust the upper body support pad height, loosen knob, depress spring button and adjust to desired position. Tighten securely.

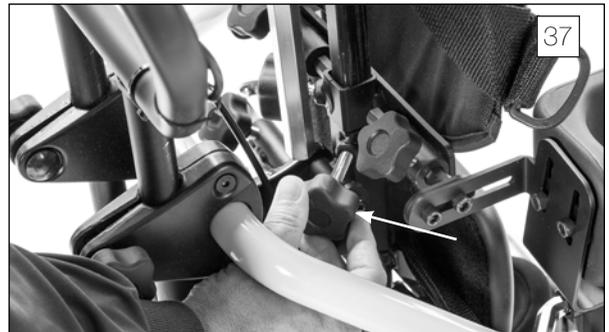


Platform Foot Plates

38. **Height**-Platform Foot Plates are only height adjustable. Loosen knob, set to desired position. Tighten securely.

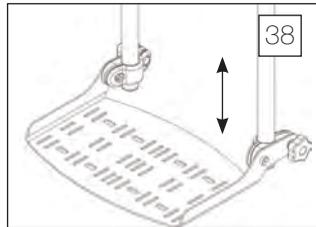


39A. **Straps**-Position straps into desired slots.

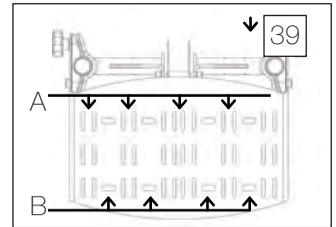


39B. **Foot Holders**-Position foot holders into desired position. Insert bolt and tighten knob securely.

Multi-Adjustable Foot Plates


40. **Height**-Loosen the ratchet handles connecting the foot bracket to vertical leg tubes to adjust foot plate height.

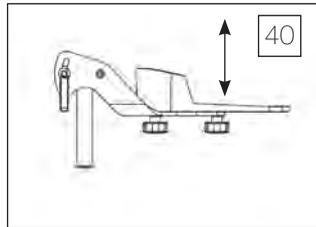


41. **Plantar/Dorsi**-Loosen ratchet handles to tilt foot plates. Tighten securely.

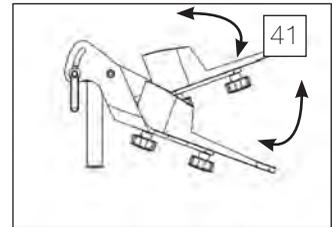


42. **Adduction**-Loosen knobs to slide foot plates together. Tighten knobs securely

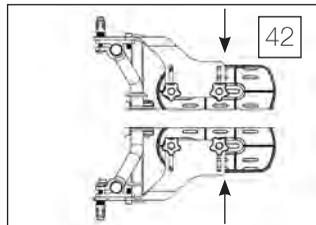


43. **Forward/Aft**-Loosen knobs underneath foot plates, slide foot plates into position. Tighten knobs securely

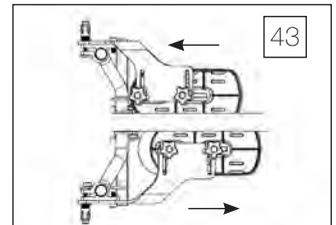

44. **Toe-in/Toe-out**-Loosen knobs underneath foot plates, slide foot plates into position. Tighten knobs securely

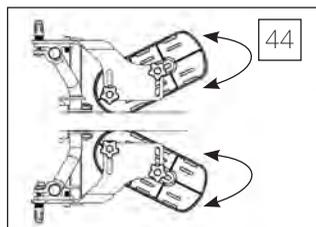

45. **Abduction**-Loosen ratchet handles to spread apart foot plates, tighten securely. Loosen knobs under foot plates and slide foot plates apart. Tighten knobs securely

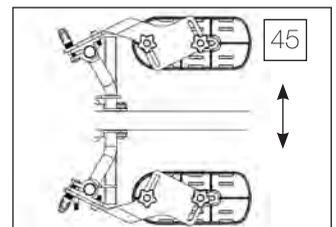

 **Caution:** Never reposition the foot plates with the child in the standing position.


 **Caution:** Avoid placing fingers in the adjustment slot.

Leg Abduction

46. Loosen knob behind the pelvic pad.



47. Abduct each leg out to the desired position up to 30 degrees per leg. Tighten securely.

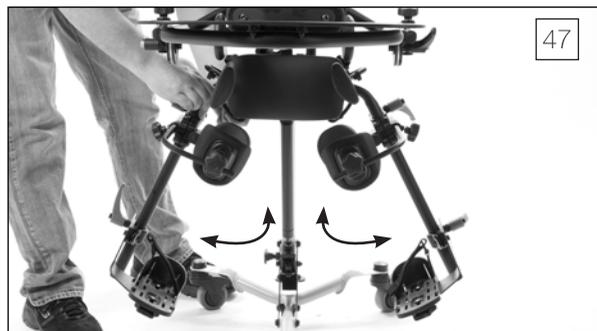

Supine (MPS only)

48. Flip the lever located behind the pelvic pad to the “supine to stand” position.

49. Adjust to the desired supine position by depressing the foot pedal and tilting. This can be done with the user in the unit.



Prone (MPS only)

50. Flip the lever located behind the pad to the “prone” position.


51. Adjust to the desired prone position by depressing the foot pedal and tilting. This can be done with the user in the unit.



Dual Control

52. Squeeze the hand lever to place the unit in the desired position.

53. Use the foot pedal to place the unit in the desired position.

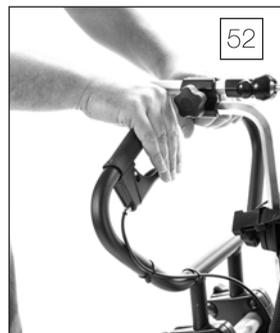

Gas Spring Lockout

54. Pull the lockout knob out and twist to unlock. Your unit can now be positioned using the foot pedal or hand lever.

55. Twist until the knob is vertical and locks into position. The gas spring is now locked from unintended movement.




Directional Locking Caster

56. Lock the right rear caster to allow the unit to guide straight. Locking the directional caster locks the directional position of the caster. This will not prevent the caster from rolling.




Parts Detail-Supine

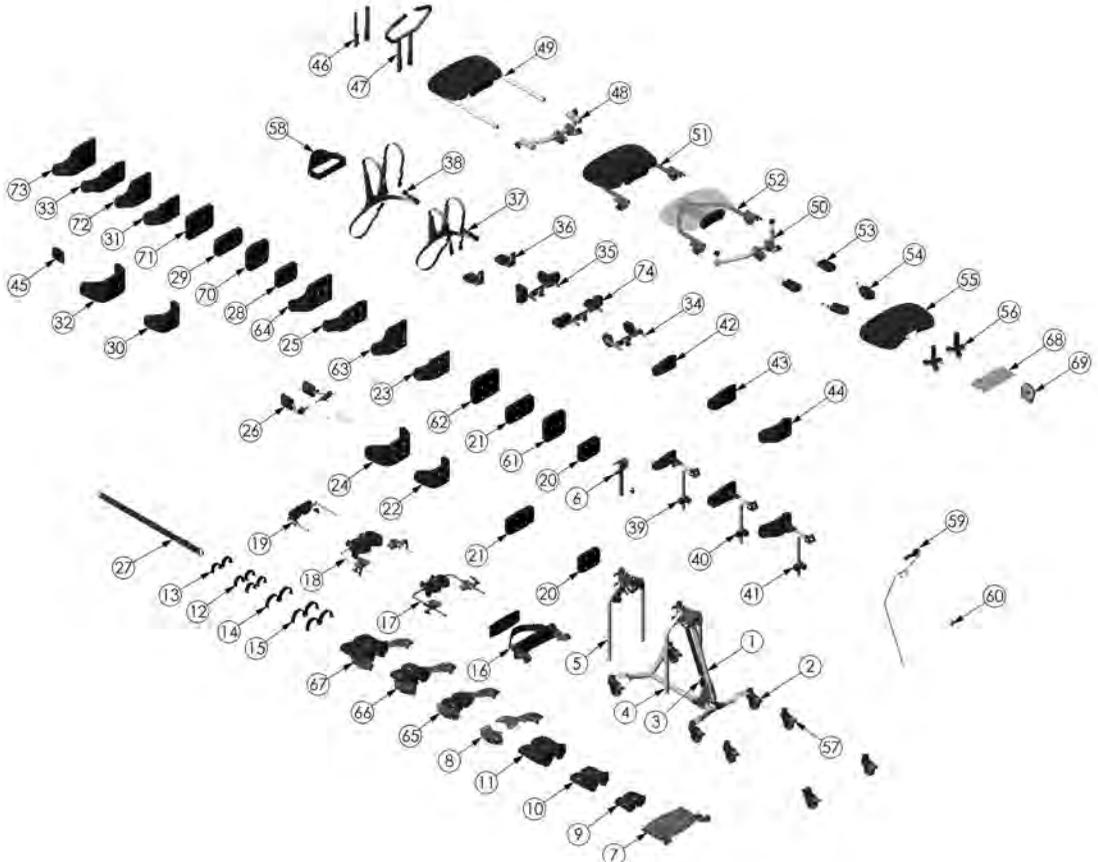

ITEM NO.	PART NO.	REV	QTY.	NAME	ITEM NO.	PART NO.	REV	QTY.	NAME
1	A5522	000	1	ASMBLY BASE ZING SUPINE	38	T50088	000	1	ASMBLY CHEST VEST SM
2	A5526	000	1	ASMBLY CASTERS BASIC MPS	39	Y5626	A	1	ASMBLY HEAD SUPPORT XS
3	A5526	000	1	ASMBLY GAS SPRING LIFT MPS	40	Y5628	A	1	ASMBLY HEAD SUPPORT SMALL
4	A5530	000	1	ASMBLY LEGS MPS STANDARD	41	A5586	000	1	ASMBLY HEAD REST CONTOUR SM
5	A5532	000	1	ASMBLY LEGS MPS ABDUCTION	42	A3035	000	1	SLIP COVER HEAD REST XS
6	A5534	000	1	ASMBLY MAIST MPS	43	A3034	000	1	SLIP COVER HEAD REST SM
7	A5534	000	1	ASMBLY PLATFORM FOOT PLATE MPS	44	A3037	000	1	COVER HEAD REST CONTOUR SM
8	A5538	000	1	ASMBLY FOOT PLATE MPS MULTI-ADJ	45	A3039	000	1	SLIP COVER 3.5" LATERAL PAD
9	A5540	000	1	ASMBLY FOOT PLATE MPS EI	46	A5592	000	1	ASMBLY TRAY ASSEMBLY MOUNT MPS
10	A5542	000	1	ASMBLY FOOT PLATES XS	47	A5594	000	1	ASMBLY PUSH HANDLE MPS
11	A5544	000	1	ASMBLY FOOT PLATES SM	48	A5595	000	1	ASMBLY TABLE MOUNT BASIC MPS
12	A5546	000	1	ASMBLY SECURE FOOT STRAPS EI	49	A5596	000	1	ASMBLY TRAY MOLDED MPS
13	A5548	000	1	ASMBLY FOOT STRAPS EI	50	A5599	000	1	ASMBLY TABLE MOUNT TILT MPS
14	T50080	A	1	ASMBLY FOOT STRAP PED	51	A5600	000	1	ASMBLY TRAY SWING AWAY MPS
15	T50082	A	1	ASMBLY FOOT STRAP SECURE PED	52	A5602	000	1	ASMBLY TRAY SWINGS AWAY CLEAR MPS
16	A5550	000	1	ASMBLY KNEE PAD SOFT MPS	53	A5604	000	1	ASMBLY FOREARM WINGS MPS
17	A5554	000	1	ASMBLY KNEE MULTI ADJUSTABLE EI	54	A5604	000	1	ASMBLY FOREARM WINGS CLEAR MPS
18	A5558	000	1	ASMBLY KNEE MULTI-ADJUSTABLE XS	55	A3038	000	1	PADDED SLIP COVER TRAY MPS
19	A5560	000	1	ASMBLY CALF PAD INDEPENDENT MPS	56	NG50042	000	1	SUPPORT GRIP OPTION
20	A5562	000	1	ASMBLY PAD PLANAR 5" X 7"	57	A5608	000	1	ASMBLY CASTERS DIRECTIONAL MPS
21	A5564	000	1	ASMBLY PAD PLANAR 5" X 9"	58	A5610	000	1	ASMBLY CHEST STRAP MPS
22	A5566	000	1	ASMBLY CONTOUR PELVIC 5" EI	59	A5612	000	1	ASMBLY DUAL LIFT CONTROL MPS XS
23	A5568	000	1	ASMBLY CONTOUR BACK 5" EI	60	A5614	000	1	ASMBLY GAS SPRING LOCK OUT
24	A5570	000	1	ASMBLY CONTOUR PELVIC 5" XS	61	A5616	000	1	ASMBLY PAD PLANAR 7" X 7"
25	A5572	000	1	ASMBLY CONTOUR BACK 5" XS	62	A5618	000	1	ASMBLY PAD PLANAR 7" X 9"
26	A5574	000	1	ASMBLY HIP GUIDES MPS XS	63	A5620	000	1	ASMBLY CONTOUR BACK 7" EI
27	A3011	000	1	POSITIONING BELT ADJUSTABLE EI	64	A5622	000	1	ASMBLY CONTOUR BACK 7" XS
28	A3028	000	1	SLIP COVER PLANAR 5" X 7"	65	A5624	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS EI
29	A3029	000	1	SLIP COVER PLANAR 5" X 9"	66	A5626	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS XS
30	A3030	000	1	SLIP COVER CONTOUR PELVIC 5" EI	67	A5628	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS SM
31	A3031	000	1	SLIP COVER CONTOUR BACK 5" EI	68	80809	A	1	TOOL POUCH
32	A3032	000	1	SLIP COVER CONTOUR PELVIC 5" XS	69	80246	000	1	SEAT ANGLE LOCATOR
33	A3033	000	1	SLIP COVER CONTOUR BACK 5" XS	70	A3045	000	1	SLIP COVER PLANAR 7" X 7"
34	A5574	000	1	ASMBLY LATERAL GUIDES MPS XS	71	A3046	000	1	SLIP COVER PLANAR 7" X 9"
35	A5578	000	1	ASMBLY LATERAL W/ ELBOW PAD MPS XS	72	A3047	000	1	SLIP COVER CONTOUR BACK 7" EI
36	A5580	000	1	ASMBLY ELBOW STOP W/ ARM REST	73	A3048	000	1	SLIP COVER CONTOUR BACK 7" XS
37	T50086	000	1	ASMBLY CHEST VEST XS	74	A5630	000	1	ASMBLY ELBOW/ ARM PAD MPS XS

Parts Detail-MPS (multi-position stander)

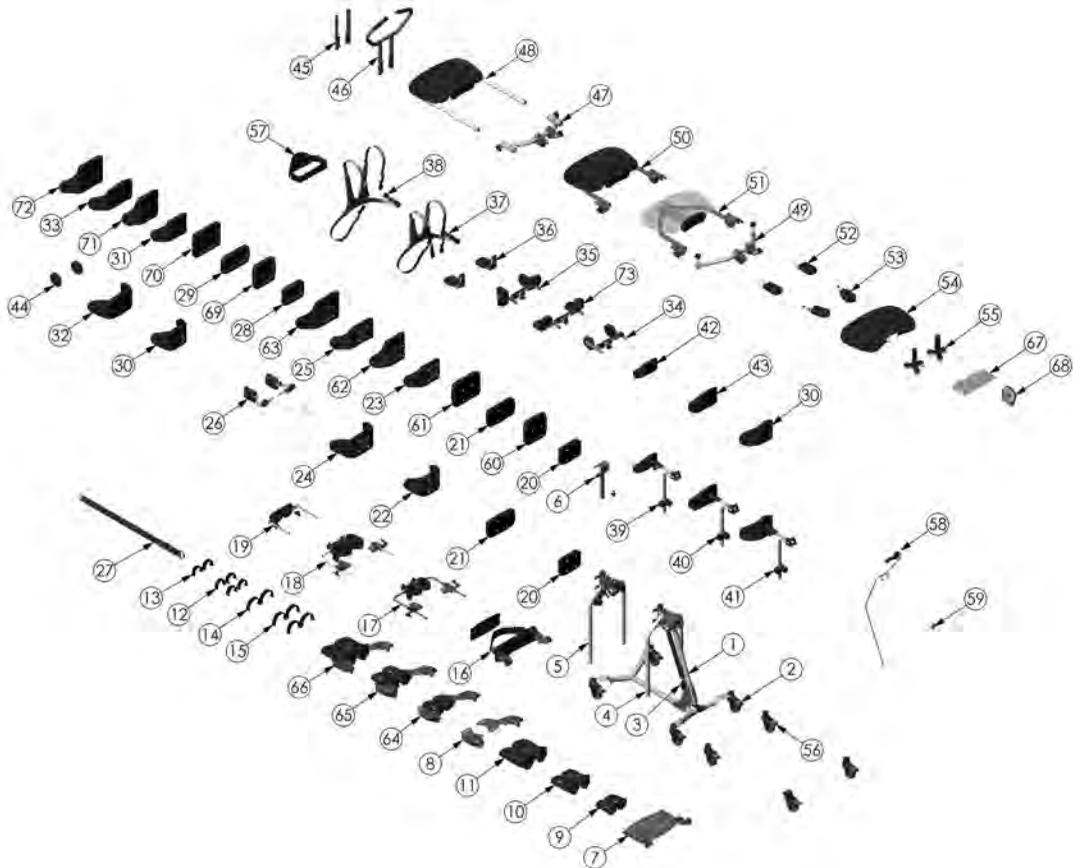

ITEM NO.	PART NO.	REV	QTY.	NAME	ITEM NO.	PART NO.	REV	QTY.	NAME
1	A5520	000	1	ASMBLY BASE ZING FORWARD PRONE	38	T50088	000	1	ASMBLY CHEST VEST SM
2	A5526	000	1	ASMBLY CASTERS BASIC MPS	39	Y5626	A	1	ASMBLY HEAD SUPPORT XS
3	A5526	000	1	ASMBLY GAS SPRING LIFT MPS	40	Y5626	A	1	ASMBLY HEAD SUPPORT SMALL
4	A5530	000	1	ASMBLY LEGS MPS STANDARD	41	A5586	000	1	ASMBLY HEAD REST CONTOUR SM
5	A5532	000	1	ASMBLY LEGS MPS ABDUCTION	42	A3035	000	1	SLIP COVER HEAD REST XS
6	A5534	000	1	ASMBLY MAST MPS	43	A3036	000	1	SLIP COVER HEAD REST SM
7	A5534	000	1	ASMBLY PLATFORM FOOT PLATE MPS	44	A3037	000	1	COVER HEAD REST CONTOUR SM
8	A5538	000	1	ASMBLY FOOT PLATE MPS MULTI-ADJ	45	A3039	000	1	SLIP COVER 3.5" LATERAL PAD
9	A5540	000	1	ASMBLY FOOT PLATE MPS EI	46	A5592	000	1	ASMBLY TRAY ASSEMBLY MOUNT MPS
10	A5542	000	1	ASMBLY FOOT PLATES XS	47	A5594	000	1	ASMBLY PUSH HANDLE MPS
11	A5544	000	1	ASMBLY FOOT PLATES SM	48	A5595	000	1	ASMBLY TABLE MOUNT BASIC MPS
12	A5546	000	1	ASMBLY SECURE FOOT STRAPS EI	49	A5596	000	1	ASMBLY TRAY MOULDED MPS
13	A5548	000	1	ASMBLY FOOT STRAPS EI	50	A5599	000	1	ASMBLY TABLE MOUNT TILT MPS
14	T50080	A	1	ASMBLY FOOT STRAP PED	51	A5600	000	1	ASMBLY TRAY SWING AWAY MPS
15	T50082	A	1	ASMBLY FOOT STRAP SECURE PED	52	A5602	000	1	ASMBLY TRAY SWING AWAY CLEAR MPS
16	A5550	000	1	ASMBLY KNEE PAD SOFT MPS	53	A5604	000	1	ASMBLY FOREARM WINGS MPS
17	A5556	000	1	ASMBLY KNEE MULTI ADJUSTABLE EI	54	A5606	000	1	ASMBLY FOREARM WINGS CLEAR MPS
18	A5556	000	1	ASMBLY KNEE MULTI-ADJUSTABLE XS	55	A3038	000	1	PADDED SLIP COVER TRAY MPS
19	A5560	000	1	ASMBLY CALF PAD INDEPENDENT MPS	56	NG50042	000	1	SUPPORT GRIP OPTION
20	A5562	000	1	ASMBLY PAD PLANAR 5" X 7"	57	A5608	000	1	ASMBLY CASTERS DIRECTIONAL MPS
21	A5564	000	1	ASMBLY PAD PLANAR 5" X 9"	58	A5610	000	1	ASMBLY CHEST STRAP MPS
22	A5566	000	1	ASMBLY CONTOUR PELVIC 5" EI	59	A5612	000	1	ASMBLY DUAL LIFT CONTROL MPS XS
23	A5566	000	1	ASMBLY CONTOUR BACK 5" EI	60	A5614	000	1	ASMBLY GAS SPRING LOCK OUT
24	A5570	000	1	ASMBLY CONTOUR PELVIC 5" XS	61	A5616	000	1	ASMBLY PAD PLANAR 7" X 7"
25	A5572	000	1	ASMBLY CONTOUR BACK 5" XS	62	A5618	000	1	ASMBLY PAD PLANAR 7" X 9"
26	A5574	000	1	ASMBLY HIP GUIDES MPS XS	63	A5620	000	1	ASMBLY CONTOUR BACK 7" EI
27	A3011	000	1	POSITIONING BELT ADJUSTABLE EI	64	A6422	000	1	ASMBLY CONTOUR BACK 7" XS
28	A3028	000	1	SLIP COVER PLANAR 5" X 7"	65	A5624	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS EI
29	A3029	000	1	SLIP COVER PLANAR 5" X 9"	66	A5626	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS XS
30	A3030	000	1	SLIP COVER CONTOUR PELVIC 5" EI	67	A5628	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS SM
31	A3031	000	1	SLIP COVER CONTOUR BACK 5" EI	68	B0609	A	1	TOOL POUCH
32	A3032	000	1	SLIP COVER CONTOUR BACK 5" XS	69	B0246	000	1	SEAT ANGLE LOCATOR
33	A3033	000	1	SLIP COVER CONTOUR BACK 6" XS	70	A3045	000	1	SLIP COVER PLANAR 7" X 7"
34	A5574	000	1	ASMBLY LATERAL GUIDES MPS XS	71	A3046	000	1	SLIP COVER PLANAR 7" X 9"
35	A5578	000	1	ASMBLY LATERAL W/ ELBOW PAD MPS XS	72	A3047	000	1	SLIP COVER CONTOUR BACK 7" EI
36	A5580	000	1	ASMBLY ELBOW STOP W/ARM REST	73	A3048	000	1	SLIP COVER CONTOUR BACK 7" XS
37	T50086	000	1	ASMBLY CHEST VEST XS	74	A5630	000	1	ASMBLY ELBOW/ ARM PAD MPS XS

Maintenance

Cleaning/Disinfecting

- Clean the EasyStand with a mild, non-abrasive disinfectant or soap and water.
- Check the EasyStand at least weekly to ensure all nuts, bolts and adjustable parts are tightened securely, failure to do so may result in parts falling off which may cause a choking hazard for children.
- Do not use bleach or any other bleach-based cleaners on the EasyStand.
- Disinfecting wipes are safe to use on metal and non-porous surfaces.
- Use a damp sponge with mild soap or a mild, non-abrasive disinfectant.
- Spot clean Neoprene with mild soap and water.
- Hygienic covers may be removed and hand washed with water. Air dry the covers. Always use care when removing and replacing the covers.
- Do not dry clean any upholstery items.
- If the EasyStand is being used with one user, clean the EasyStand on a monthly basis or as needed.
- If the EasyStand is being used in a multiple-user environment, the EasyStand should be cleaned after each use.

Urethane Coated Fabrics and Hygienic Washable Covers (Dartex or generic equivalent)

- The recommended cleaning and disinfection instructions should be followed whenever possible to avoid damage to the PU membrane which could lead to premature failure of the cover.
- For superficial dirt, use a disposable wipe and a warm solution of neutral detergent. Do not use abrasive cleaners.
- Disinfect using a 0.1% sodium hypochlorite solution (1000ppm chlorine), stronger bleach solutions not recommended.
- Thoroughly rinse any remaining cleaning solution with water and dry fabric before re-use or storage.
- Do not iron.

Limited Warranty

This warranty is extended only to the original purchaser/customer (or supplier non consumer who does not buy or resell). Altimate Medical, Inc., warrants the EasyStand against defects in materials and workmanship as listed below.

- Steel Frames - 5 year warranty
- Upholstered components, plastic pads, rubber parts, painted surfaces, bearings and other parts not specifically identified above - 90 day warranty
- Gas Spring - 2 year warranty

The warranty period commences for the consumer on the purchase date from the seller/supplier. If the product is rented or otherwise not sold to a consumer, the warranty period commences from the date of invoice from Altimate Medical. Any product proven to Altimate Medical satisfaction to be defective and within warranty period, shall be repaired or replaced at Altimate Medical's option.

ALTIMATE MEDICAL SOLE OBLIGATION AND YOUR EXCLUSIVE REMEDY UNDER THIS WARRANTY SHALL BE LIMITED TO SUCH REPAIR AND/OR REPLACEMENT. This warranty does not include any labor charges incurred in replacement parts installation. Freight charges to factory are at the expense of consumer to seller. Return freight charges will be prepaid by Altimate Medical. For warranty service, please contact the authorized supplier from whom you purchased your Altimate Medical product. In the event you do not receive satisfactory warranty service, please write directly to Altimate Medical at the address below. Do not return products to our factory without prior authorization.

LIMITATIONS AND EXCLUSIONS: The foregoing warranty shall not apply to serial numbered products if the serial number has been removed or defaced. Products subject to negligence, abuse, misuse, improper operation, improper maintenance, improper storage, or damages beyond Altimate Medical's control are not covered by this warranty, and that evaluation will be solely determined by Altimate Medical. Products modified without Altimate Medical's express written consent (including, but not limited to, modification through the use of unauthorized parts or attachments) are not covered by this warranty. The warranty shall not apply to problems arising from normal wear or failure to follow instructions. The warranty will be void if parts not manufactured by Altimate Medical or parts not complying with original equipment specifications are added to an Altimate Medical product.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES. ALTIMATE MEDICAL SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Altimate Medical maintains a policy of continual product improvement and reserves the right to change features, specifications, and prices without prior notification. Check with Altimate Medical for latest information.

Environmental Issues

After life of the product:

- Recycle in accordance with the recycling codes of your country.
- Recycle all metal parts.
- Plastic parts should be disposed of or recycled according to the directive in your country.
- Please inquire at your local disposal management and recycling center for local codes.
- Return gas spring cylinder to Altimate Medical, Inc. if no local recycling is available.