


Swift Commode Swift Freestanding

etac®


Manual

78323G 2013-05-17

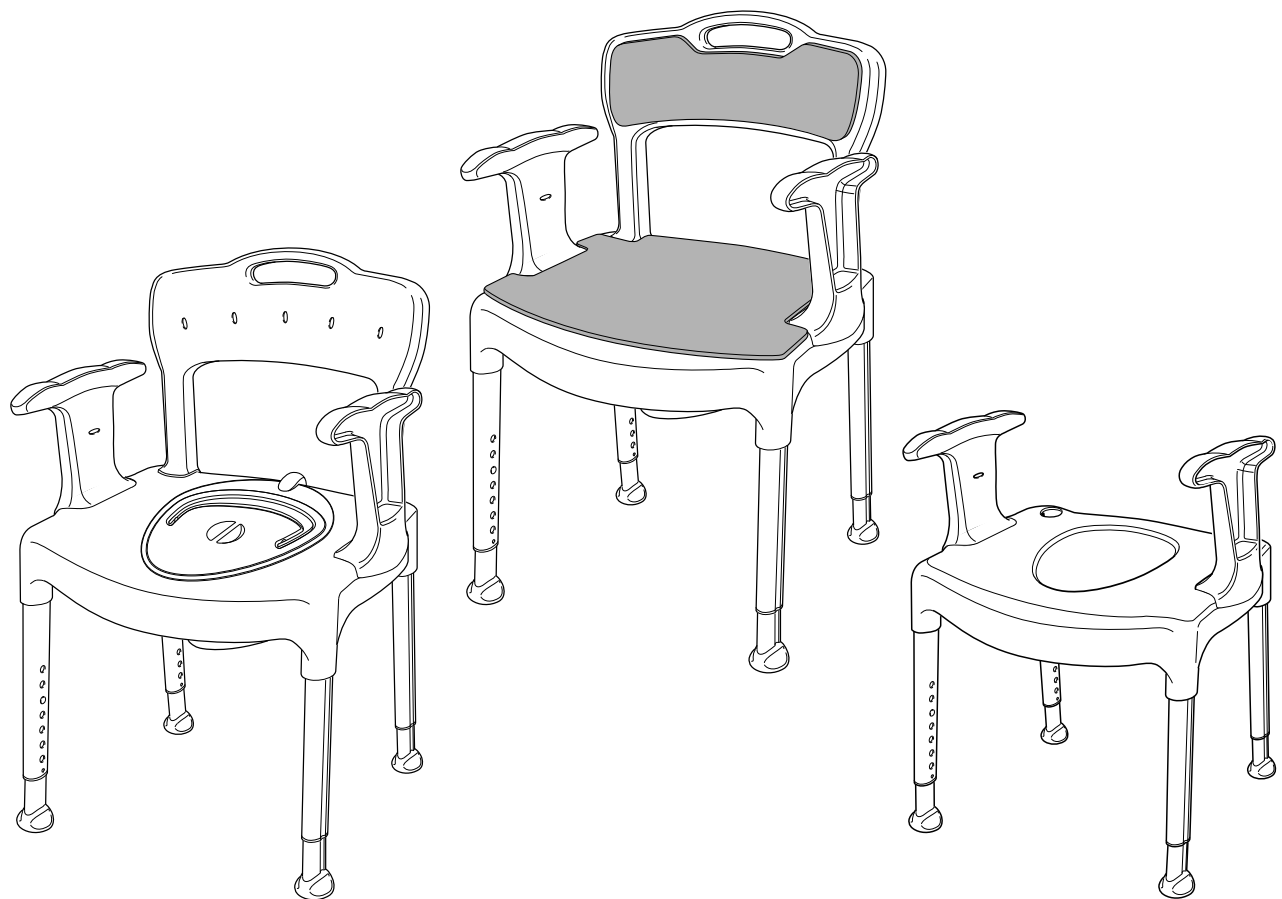


Mounting Swift


1

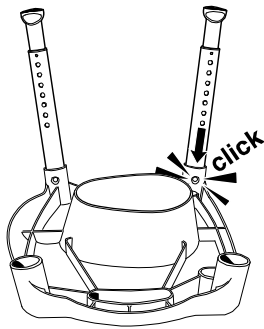

2

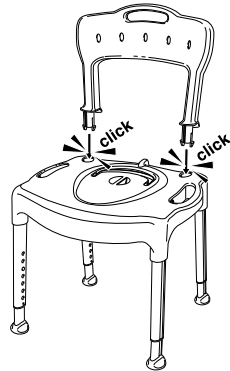


3

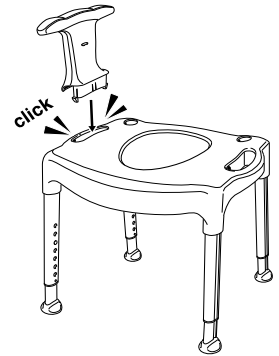


Dismounting




1

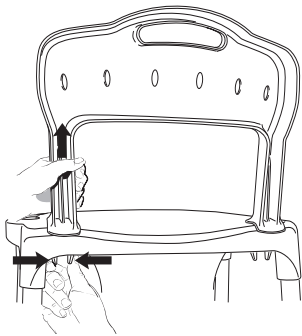



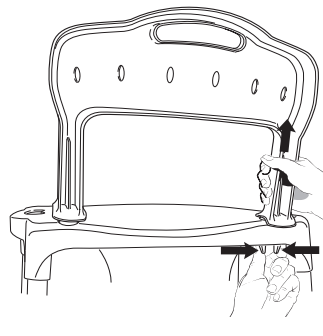





2

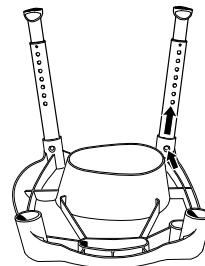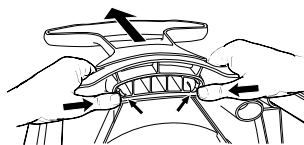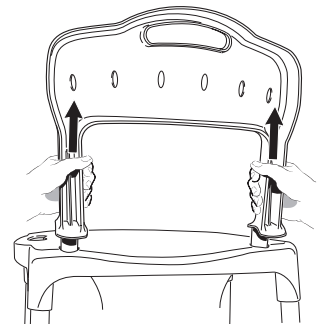

3



Technical data

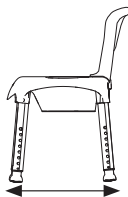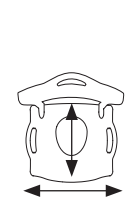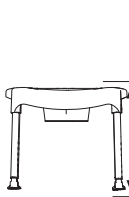								
Etac Swift Commode	210 mm	450 mm	540 mm	800-950 mm	530-550 mm	540 x 410 mm	420-570 mm	5,4 kg
Etac Swift Freestanding	210 mm	450 mm	540 mm	800-950 mm	530-550 mm	540 x 410 mm	420-570 mm	3,9 kg
Etac Swift Commode low	210 mm	450 mm	540 mm	725-800 mm	520-530 mm	540 x 410 mm	345-420 mm	5,2 kg

English

Intended use

The Swift commode is a comfortable and mobile toilet chair which makes it easier for users to go to the toilet at all times. The freestanding Swift toilet chair offers users stable seating at the correct height above the toilet seat.

Description

The products have a stable design with built-in flexibility which adapt to uneven floors. All products are fitted with height-adjustable legs that are simple to attach. A correct height setting and the use of armrests makes it easier for users to stand up. The leg ferrules offer good grip and the anti-slip pattern of the seat offer the important safety and stability properties that make the user feel safe. The Etac Swift Freestanding toilet seat raiser can be upgraded to a Swift Commode using accessories.

Mounting

Swift is easy to assemble. No tools are needed.

Accessories

Etac Swift Commode/Freestanding

Swift Commode pan including lid: portable pan for the toilet seat.

Back support: Can be easily fitted to offer users better stability.

Seat pad: A seat pad that provides both softness, warmth and friction.

Backrest pad: A back pad that provides softness, warmth and friction.

Soap dish/Lid support: A compartment that can be attached to the side of the chair to place soap, the bed pan lid, toilet paper or other items that must be at hand when one uses the Swift Commode/freestanding toilet chair. Shower head holder is included.

Safety

Warning: A defective product must not be used.

Warning: A height difference of more than one hole is not allowed for chair legs.

Warning: When moving sideways, do not lean on just one armrest.

Max user weight: Swift Commode/ Freestanding 130 kg.

Cleaning: Clean the product using a solvent-free cleaning agent with a pH value of 5-9 or with a 70% disinfectant solution.

Decontamination: Can be decontaminated at max. 85° C/185 °F for 3 minutes if required.

