



# INSTRUCTIONS

## **HOW TO USE**

1. Locate the EZ-SHOWER Bedside Shower, dowel, and string.
2. Insert dowel into top of Shower.
3. Insert string through dowel, and then tie string in knot.
4. Fill Shower with warm water. Release clamp to allow water flow; tighten clamp to stop.
5. Drain Shower completely after each use.

## **STORAGE & CLEANING**

Product is made of PVC and contains an anti-microbial agent. Rinse with warm water and mild soap, then allow to air dry thoroughly before folding and or storing.



If you have concerns about allergies or how the recommended cleaning solution may affect your health, please consult with your physician or other healthcare professional before using so he/she can recommend alternatives or, product can be disposed of after each use.