384X178
Hantag to be printed with 4 Color High Gloss Ink plus UV coating on 80 LB. White Stock.
Red guides are to indicate folds and layout only. They do not print.
Score all folds
Drill holes where indicated, 4mm
Use a CLEAR 25.4 MM round perforated waffer to seal hangtag

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LUMEX
A Graham-Field Family Brand

Features

- Soft, cushion grip reduces hand stress and fatigue
- Leg and forearm sections can be adjusted independently for a custom fit
- Vinyl-coated arm cuffs can be shaped to the user's arm for extra comfort
- Extra-large, non-skid crutch tips provide added traction and stability
- Easy push-button height adjustment with locking collet for added security
- Grip-to-floor height adjusts in 1" increments
- Arm cuff to grip height adjusts in 1" increments
- Maximum Weight Capacity 300 lb - **EVENLY DISTRIBUTED**

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Follow us on [in](#) [YouTube](#) [f](#)

© 2017, GF Health Products, Inc. All Rights Reserved. Lumex is a trademark of GF Health Products, Inc. GF Health Products, Inc. is an ISO 13485 Certified Company.

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graham field

Limited Warranty

The following warranty information is a **SUMMARY** of the complete Product Warranty provided by GF Health Products, Inc.

GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for three (3) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

EXCLUSIONS
 The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER
 THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

Outside

Instructions For Use – Please Read All Instructions Before Use.

The chart below can be used as a guide to select height-appropriate crutches. Please consult a healthcare professional for assistance with proper height adjustment.

| Item Number | Size | User Height | Adjustment Range / Handgrip to Floor |
|-------------|-------|--------------|--------------------------------------|
| 6350T | Tall | 5'5" – 6'4" | 33" – 42" |
| 6350A | Adult | 5'0" – 6'0" | 28" – 37" |
| 6350Y | Youth | 4'10" – 5'8" | 22" – 31" |

Adjusting Crutch Height

- Loosen locking collet by twisting counter-clockwise.
- Adjust crutch height: Depress push button and slide lower section upward or downward to desired height.
- Secure locking collet by turning clockwise until tight.

NOTE: An audible "click" will be heard when push button securely protrudes through the adjustment hole.

Adjust Cuff Height

- Loosen locking collet by twisting counter-clockwise.
- Adjust crutch height: Depress push button and slide lower section upward or downward to desired height.
- Secure locking collet by turning clockwise until tight.
- Arm cuffs are adjustable; adjust both cuffs for proper fit.

Maintenance

- Wipe crutches weekly with a damp cloth.
- Grip and underarm pad may be cleaned with mild soap or detergent. DO NOT use organic solvents, such as acetone, which may damage grip.
- Never use abrasive cleaners or scouring pads on any part of the crutches.
- Check hardware and positioning weekly for tightness.
- Replace crutch tips as needed.
- To guarantee the best service from your crutches, always specify Lumex replacement parts, available from your GF Health Products distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online

Safety Guidelines – Please Read Before Use

- WARNING:** This product should not be used without proper instruction from a healthcare professional.
- WARNING:** This product has a maximum user weight limitation of 300 lb.
- WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- WARNING:** If components are damaged or missing, contact your distributor immediately. DO NOT use substitute parts.
- WARNING:** Before use, ensure that crutch tips are not ripped, worn or missing and are properly seated and straight on each crutch.
- WARNING:** Always use caution when using this product on wet or slippery surfaces. DO NOT use crutches on stairs or steep inclines.
- WARNING:** Inspect crutches weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
- WARNING:** Before use, ensure that push buttons protrude fully through adjustment holes.
- WARNING:** Notice for California Customers
 California Proposition 65 **WARNING:** This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.
- WARNING:** Wing nuts **MUST** be tight at all times.

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