

Tools required: adjustable wrench

- ⚠ **WARNING: Important! Read and understand these instructions before assembling or using the IV Pole. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the IV Pole.**
- ⚠ **WARNING: Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**

### **ASSEMBLY**

#### **Unpacking**

1. Check for any obvious damage to the carton or its contents. If damage is evident, notify the carrier or your authorized Graham-Field distributor.
2. Remove all loose packing from the carton.
3. Carefully remove all the components from the carton.

#### **IV Pole Assembly**

1. Install caster base with guide pin on caster base with two holes, placing guide pin in the smaller of the two holes as shown at right.
2. Install flat washer on bolt. Install bolt through center of both caster bases into bottom of lower pole.
3. Use wrench to tighten bolt until securely fastened in lower pole.
4. Install hook assembly on upper pole. Tighten by hand until secure.


- ⚠ **WARNING: Ensure that IV Pole is assembled as described before use.**

### **OPERATION**

#### **Height Adjustment**

1. Turn collar counter-clockwise enough to free upper pole to slide up and down in lower pole. If anything is attached to hook assembly, ensure that it does not fall.
2. Adjust upper pole to desired height.
3. Turn collar clockwise until upper pole is secure in lower pole.

- ⚠ **WARNING: POTENTIAL EYE HAZARD. Maintain a safe distance from IV hooks.**

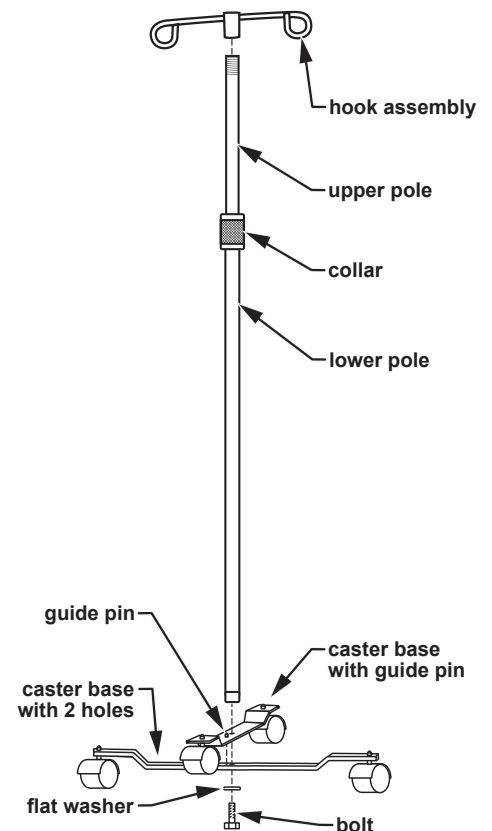

GF7012-1 IV Pole

## **MAINTENANCE**

Check the fit and tightness frequently of all fasteners to ensure that the IV Pole is securely assembled.

### **Cleaning**

Regular cleaning is recommended. To clean:

1. Gently wipe the IV Pole with a soft, clean cloth dampened with water and a mild NON-ABRASIVE detergent (household cleaner, soapy water, or hospital grade disinfectant).
2. Rinse IV Pole with a soft, clean cloth dampened with water.
3. Dry IV Pole with a clean, soft cloth.

**▲ NOTICE: Do not wash under water pressure, steam clean, or use cleansers that could damage the IV Pole's finish.**

## **THREE-YEAR LIMITED WARRANTY**

### **SCOPE OF WARRANTY**

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted.

This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable.

The warranted components and time period are set forth below:

GF7012-1 IV Pole: three years

The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

### **OBTAINING WARRANTY SERVICE**

A GF Customer Service Representative must authorize warranty service. Please contact the GF Customer Service department by calling \_\_\_\_\_, sending a fax request to \_\_\_\_\_ or by e-mailing a request to \_\_\_\_\_. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

### **EXCLUSIONS**

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

### **ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER**

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS.

THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

### **NOTES:**

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the Bill of Lading and must be made with immediacy. The ICC regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within thirty (30) days of the invoice date.



GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, including the most current version of these instructions, please visit our website at \_\_\_\_\_. **Graham-Field** and **Lumex** are registered trademarks of GF Health Products, Inc.

© 2010 GF Health Products, Inc.