



Imperial Collection Commode

Colección Imperial de Inodoro
Items 6438A, 7446A-2

Features

- Arms drop independently below seat level for lateral transfers (6438A only)
- Arms automatically lock in raised position
- Extra wide platform seat is comfortable for long periods of sitting
- Welded steel construction and extra wide base provide added stability
- Cushioned armrests enhance user comfort
- Comes complete with commode pail, cover and splash guard
- Removable back bar facilitates use as a bedside commode, raised toilet seat or toilet safety frame
- 6438A – 600lb maximum weight capacity **EVENLY DISTRIBUTED**
- 7446A-2 – 400lb maximum weight capacity **EVENLY DISTRIBUTED**
- Designed to meet the requirements of HCPCS code: E0168

Características

- Apoyabrazos se bajan independiente abajo del nivel del asiento para transferir lateral con seguridad (solamente 6438A)
- Apoyabrazos se cierran automáticamente en la posición subida
- Asiento plataforma mas ancho es comfortable para sentarse por periodos largos
- Construcción de acero aumenta esfuerzo y estabilidad
- Apoyabrazos acolchonados aumentan comodidad de usuario
- Viene completa con el balde de inodoro, tapa y salpicadera
- Respaldo desmontable facilita uso como inodoro al lado de cama, asiento elevado o marco de inodoro para seguridad.
- 6438A – Capacidad de peso máximo de 600 lb **DISTRIBUIDO UNIFORMEMENTE**
- 7446A-2 – Capacidad de peso máximo de 400 lb **DISTRIBUIDO UNIFORMEMENTE**
- Diseñado a reunir los requisitos de código de HCPCS: E0168

GF0900148RevB14



Item 6438A

User Instructions and Warranty Information

Important: Read Before Use

Instrucciones e Informacion de la Garantia

Importante: Lea Antes de Uso



Instructions for Use – Please Read All Instructions Before Use.

Setup


1. Remove frame, commode platform and back support from box. Unwrap all packaging materials.
2. Commode platform is already attached to frame, as shown below; ensure that all sides of platform are secured to frame.
3. The commode comes in its lowest position. Adjusting to desired height is recommended before use.

Arm Adjustment

- **Drop Armrest:** to drop the armrest, push the release lever inwards to release the armrest. This releases the armrest allowing for the transfer on and off the commode.
- **Raise Armrest:** to raise the armrest, lift the armrest to the upright position, ensuring that the push button protrudes fully from locking hole.

Adjusting Leg Height

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole.
NOTE: An audible "click" will be heard.
2. Repeat for other 3 legs.
CAUTION: Ensure that all four commode legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

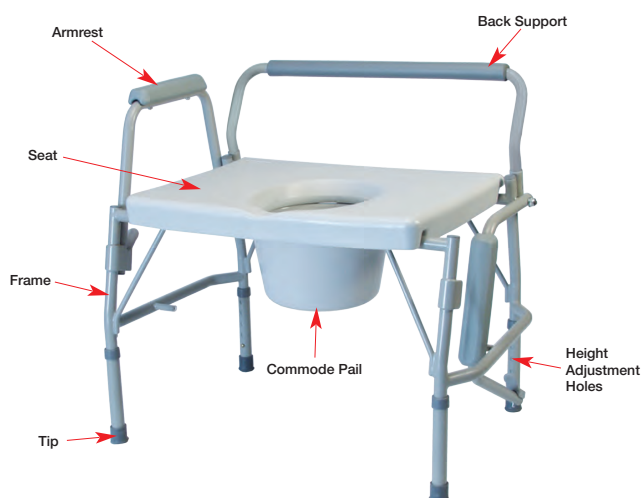

Maintenance

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc authorized distributor.

Safety Guidelines – Please Read Before Use

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

WARNING

WARNING: DO NOT use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

WARNING: Users with limited physical capabilities should be supervised or assisted when using the commode.

WARNING: Item 6438A has a maximum weight capacity of 600 lb. **EVENLY DISTRIBUTED**
Item 7446A-2 has a maximum weight capacity of 400 lb. **EVENLY DISTRIBUTED**

WARNING: GF Health Products Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly, or use of this product.

WARNING: Leg extensions with rubber tips **MUST** be in contact with the floor at all times.

WARNING: Ensure that the push buttons fully protrude through the same height adjustment hole of each leg extension. This ensures that the leg extensions are securely locked in position and that an even height adjustment is achieved.

WARNING: Make sure that all attaching hardware, screws, nuts and/or bolts are tight at all times.

WARNING: Inspect rubber tips on the leg extensions for rips, tears, cracks or wear or if they are missing. Replace them **IMMEDIATELY** if any of these conditions exist.

WARNING: The commode seat **MUST** be installed before sitting on the commode.

WARNING: Notice for California Customers
California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

WARNING: If components are damaged or missing, contact your GF Health Products Inc. distributor immediately.