



# Reclining PVC Shower Chair

Items: 89333, 89337, 89339

## Contents:

Partially assembled (reclined) shower chair  
Four (4) caster wheels (2 braking & 2 non-braking)  
PVC high-back with mesh sling  
PVC push/pull handle  
Twelve (12) ½" stainless steel screws

## Assembly Instructions:

### Step 1

Remove partially assembled shower chair from shipping container/box.

### Step 2

Insert the 4 casters into the opening of the fittings. Secure the casters by inserting and tightening ½" screws into the pre-drilled pilot holes on the fittings with a Phillips screwdriver.

### Step 3

The sliding footrest goes directly under the commode seat on the horizontal bars. Snap the sliding footrest onto each side and secure the vinyl strap onto each fitting. Once the straps are secure, the sliding footrest should be able to function and a household furniture polish can help the footrest slide easier.

### Step 4

Insert each arm into the fittings located directly on each side of the toilet commode seat. Secure each section of the arm with ½" stainless steel screws (2 per side).

### Step 5

Insert the PVC high-back with mesh sling into the opening of the fittings. Secure with ½" screws.

### Step 6

Insert the push/pull handle into the rear top section of the reclining chair. The push/pull handle also serves as the safety catch for the reclining back.



**NOTE:** ALL ATTACHED SECTIONS SHOULD BE DOUBLE CHECKED FOR PROPER INSTALLMENT.