



- Lightweight, durable, rust-resistant frame
- Available with and without backrest
- Seat height adjusts in 1" increments for various user heights
- Wide seat with handles (7932 Only) provides additional comfort
- Maximum Weight Capacity 600 lb - EVENLY DISTRIBUTED



Bariatric Bath Seat

Items 7932A & 7939A



Instructions for Use – Please Read All Instructions Before Use.

Setup

Installing the Back

(Only for 7939A)

1. Place bath seat on its side on a flat working surface.
2. Locate the channel guides (4 each) on underside of seat. Align backrest support tubes with channel guides.
3. Slide backrest seat support tubes through channel guides until seat support tube push buttons protrude fully through channel guide holes.

NOTE: Ensure seat support tube buttons protrude fully through channel guide holes.

Caution: Backrest is not designed to support the full weight of the user.

Adjusting Leg Height

Legs are already installed on the bath seat.

To adjust leg height:

1. Press push button and slide leg extension up or down to desired adjustment hole.
2. Ensure push button fully protrudes through the leg adjustment holes of the leg extension.

NOTE: An audible “click” will be heard when push button securely protrudes through the adjustment hole.

3. Repeat STEPS 1 and 2 for the other three legs.

NOTE: Ensure all four legs are adjusted to the same height and that all height adjustment buttons protrude fully through the adjustment holes before use.



Maintenance

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

Safety Guidelines – Please Read Before Use

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

WARNING

WARNING: DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

WARNING: Item has a maximum weight capacity of 600 lb. EVENLY DISTRIBUTED.

WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

WARNING: ALWAYS inspect the bath seat to ensure that it is properly positioned and stable before using. DO NOT use if bath seat is wobbly or unstable.

WARNING: If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately. DO NOT use substitute parts.

WARNING: Check leg tips for rips, tears, cracks or wear. If any of these conditions exist, replace leg tips IMMEDIATELY.

WARNING: DO NOT use this as a transfer bench or climbing device.

WARNING: All four leg tips MUST be in contact with shower/tub floor at all times.

WARNING: Ensure that the four chair legs are adjusted to the same height and that the push buttons fully protrude through their respective adjustment holes BEFORE use.

WARNING: Only use GF Health Products, Inc. accessories with this product.

WARNING: This product should only be used with tub floors wider than 18 ¾ inches.

WARNING: Users with limited physical capabilities must be supervised or assisted when using the bath seat.

WARNING: Make certain ALL screws and nuts are tight at all times.

WARNING: Notice for California Customers
California Proposition 65 **WARNING:** This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

WARNING: DO NOT submerge product in water, otherwise damage may result.