



## Bariatric Transfer Bench

Item 7925A

### Features:

- Provides assistance to those who have difficulty stepping over the bathtub walls
- Anodized aluminum frame is lightweight, durable, and rust resistant
- Drainage holes in seat minimizes water build up
- Reversible backrest to accommodate any bathroom
- Support Arm aids in lateral transfer
- Maximum Weight Capacity 600 lb - EVENLY DISTRIBUTED
- Designed to meet the requirements of HCPCS code: E0248
- Lifetime Limited Warranty



## Instructions for Use – Please Read All Instructions Before Use.

### Set-Up

1. Remove frame and back from box.
2. Place frame on side on a working surface.
3. Attach the leg extensions to the transfer bench, if not attached already. Depress the buttons and slide the legs onto the frame until the buttons protrude fully from the desired holes.

### Attaching the Backrest (if not attached already)

1. Determine the side on which the backrest should be mounted by knowing which direction the user will face when sitting on the bench in the tub.
2. Position the bench upright on a level surface.
3. Locate the channel guides (4 each) on underside of seat. Align backrest support tubes with channel guides in position appropriate to step 1.
4. Slide backrest seat support tubes through channel guides until seat support tube push buttons protrude fully through channel guide holes.

**CAUTION:** Backrest is not designed to support the full weight of the user.

### Adjusting Leg Height

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button clicks into place in the desired hole.
2. Repeat for other 3 legs.

**CAUTION:** Ensure that all four legs are adjusted to the same height.

### Using Your Transfer Bench

**NOTE:** Be sure to move slowly throughout use to prevent injuries.

**NOTE:** Children or those who need assistance should NOT be left unattended during use of this product.

#### From a seated position (such as a wheelchair):

1. Place your wheelchair next to the bench, facing the front of the bathtub.
2. Engage wheelchair locks.
3. Transfer your torso to the transfer bench first, and then lift your legs into the tub. Move slowly to prevent injuries.
4. After bathing, lift your legs out of the tub first, then your torso.

#### From a standing position:

1. Sit on the side of the transfer bench that is outside of the tub.
2. Extend your arms to grab the support arm of the bench and slide your body over onto the bench. Lift each leg over the tub slowly.
3. Position yourself to a comfortable position in center of bench.
4. After bathing, lift your legs out of the tub first, then your torso slowly.



## Maintenance

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

## Safety Guidelines – Please Read Before Use

**NOTE:** Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

## WARNING

**WARNING:** DO NOT use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

**WARNING:** Users with limited physical capabilities should be supervised or assisted when using the transfer bench.

**WARNING:** Maximum Weight Capacity 600 lb - EVENLY DISTRIBUTED

**WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

**WARNING:** Leg extensions with rubber tips MUST be in contact with the floor at all times.

**WARNING:** Make sure that all attaching hardware is tight at all times.

**WARNING:** Inspect rubber tips on the leg extensions for rips, tears, cracks or wear or if they are missing. Replace them IMMEDIATELY if any of these conditions exist.

**WARNING:** The transfer bench seat MUST be installed before sitting on the transfer bench.

**WARNING:** Notice for California Customers  
**California Proposition 65 WARNING:** This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

**WARNING:** If components are damaged or missing, contact your GF Health Products, Inc. Health Products distributor immediately. DO NOT use substitute parts.