



7910A-1/7915A-1

- Designed for transport to and from the shower
- Anodized, non-corrosive aluminum frame construction will not rust
- 5" swivel casters with rear wheel locks facilitate maneuverability in tight spaces
- Rolls easily over most toilets
- Allows complete access to body for easy cleaning during bathing
- Maximum Weight Capacity 300 lb - EVENLY DISTRIBUTED
- Lifetime Limited Warranty

7915A-1 Only

- Powder coated steel front brace with footrests
- Foot plates fold up and out of the way for unobstructed entry and exit



Shower Transport Chair

Silla de Transporte de Ducha

Items 7910A-1 & 7915A-1



Instructions for Use – Please Read All Instructions Before Use.

Attaching the Backrest (7910A-1 & 7915A-1)

1. Remove chair and back support from the box. Place chair on a flat surface.
2. Lock rear casters to prevent chair from moving during assembly.
3. Locate the channel guides (2 each) on the back side of the chair frame near the seat. Align the backrest support tubes with the channel guides.
4. Depress the push buttons on the back support tubing and slide the tubing through the channel guides until the push buttons protrude fully through the channel guide holes.

Attaching the Footrests (Only for 7915A-1)

1. Locate the footrests (2 each).
2. Insert 1 footrest into the chair leg extension.
3. Ensure footplate is able to fold down to the inside.
4. Move footrest upward or downward to desired length, ensuring alignment holes match with holes in chair leg extension.
5. Once the holes are aligned, insert a screw into the hole, rotating the screw clockwise. Repeat this step for the second screw.
6. Use a screw driver to tighten the screws.

Caution: Do not over tighten.

7. Repeat these steps for the remaining footrest.



Maintenance

- Use a mild soap and water solution, rinse with clear water, and then dry with a soft cloth. Ensure that chair is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of chair.
- Check fit and tightness of all nuts, bolts, and knobs periodically to ensure that chair is safe to use.
- To guarantee the best service from your chair, always specify Lumex replacement parts, available from your GF Health Products, Inc. distributor.

Safety Guidelines – Please Read Before Use

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

WARNING

WARNING: GF Health Products Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

WARNING: This product has a maximum weight limit of 250lb. **EVENLY DISTRIBUTED**

WARNING: Users with limited physical capabilities should be supervised or assisted when using this product.

WARNING: Casters **MUST** be in contact with the floor at all times during use.

WARNING: Make sure that all attaching hardware, screws, nuts and/or bolts are tight at all times.

WARNING: This product should not be used without proper instruction from a healthcare professional.

WARNING: Notice for California Customers
California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

WARNING: If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately.