



#### Features:

- Anodized aluminum frame supports a 300 lb weight capacity- **EVENLY DISTRIBUTED**
- Dual release folding mechanism allows frame to be folded easily and is ideal for users with limited dexterity
- Contoured, textured handgrips provide additional comfort
- Stable and lightweight frame makes it easy to maneuver



## Everyday Adult and Junior Dual-Release Folding Walker

Items 716070A, 716170A, 716270A, 716370A



### Instructions for Use – Please Read All Instructions Before Use.

#### Assembly of the Walker

1. Remove closed walker from carton.
2. Install legs by depressing the height adjustment button and slide onto walker frame until the button engages in the desired adjustment hole. Adjust all leg attachments to the same height.

**WARNING:** Ensure all push buttons protrude fully from height -adjustment holes, all legs are adjusted to the same height, and wheeled leg attachments are positioned on the front and towards the outside of the walker.

3. To open walker, push handgrips and sides out away from each other until the release mechanisms lock into place.

#### Closing the Walker

1. Push downward on release button and hold.
2. Grab one handgrip and push inwards towards center of walker.
3. Repeat for other handgrip.

#### Adjusting the Height

**NOTE:** Height adjustment should be made to desired height before operation.

Consult with healthcare professional for proper adjustment.

1. To adjust the height of the leg attachment, depress the adjustment button and slide upwards or downwards to the desired height.
2. Adjust the height of the walker by aligning all four leg attachments to the same position height.

When walking, the user should grasp one handgrip in each hand, positioning body between the two rear walker legs, and walking at a comfortable pace. The rear walker legs should be in contact with the floor at all times.



#### Maintenance

- For dust and dirt, wipe walker gently with damp, soft cloth.
- Inspect wheels (if applicable) periodically for wear or damage. Use a mild soap and water solution to clean, rinse with clear water, then dry with a soft cloth.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the walker.
- Inspect Walker weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your GF Health Products, Inc. distributor for repair and/or appropriate replacement parts.
- If handgrips are loose, DO NOT use. Contact your GF Health Products, Inc. authorized distributor.
- Replace any broken, damaged or worn items immediately.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

### Safety Guidelines – Please Read Before Use

1. Each individual should ALWAYS consult with their physician or other health care professional to determine proper adjustment and usage.
2. Before use, be sure that the walker is in its open and locked position.
3. All legs MUST be in contact with the floor at all times during use. This will ensure the walker is properly balanced.
4. This product has a maximum user weight limitation of 300 lbs - **EVENLY DISTRIBUTED**. ALWAYS observe the user weight limitation on the hangtag.
5. Ensure handgrips DO NOT twist on walker – otherwise damage or injury may occur.
6. ALWAYS test to see that the walker and attachments are properly and securely locked in the open position BEFORE using.
7. After installation and before use, ensure that all attaching hardware is securely tightened.
8. ALWAYS use caution when using the walker on wet and slippery surfaces. DO NOT use the walker on stairs and steep inclines.
9. Before attempting to reach objects or pick them up from the floor by reaching down between your knees, place feet securely on the floor. Use **EXTREME** caution when reaching for any object.
10. GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
11. Inspect Walker weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.

#### WARNING

**WARNING:** DO NOT use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

**WARNING:** If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately. DO NOT substitute parts.

**WARNING:** Before use, ensure that all assembled parts are secure and all hardware is tightened.

**WARNING:** Care should be taken to ensure that all height adjustments are secure, and that wheels (if applicable) and moving objects are in good working order before using this walker.

**WARNING:** ALWAYS observe the user weight limitation on the hangtag of this product.

**WARNING:** Notice for California Customers

**California Proposition 65 WARNING:** This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

**WARNING:** Be careful when reaching for objects when using the walker. Reaching for objects can change the weight distribution of the walker and may cause the walker to tip, resulting in injury or damage.