



## Platinum Collection Padded Bath Seat

- Lightweight, durable, rust-resistant frame
- Water-tight cushioned seat and backrest for comfort
- Available with or without support arms
- Seat height adjusts in 1" increments for various user heights
- 300 lb weight capacity **EVENLY DISTRIBUTED**

Items 7944A, 7945A, 7946A, 7947A



### Instructions for Use – Please Read All Instructions Before Use.

#### Setup

1. Remove all components from the box.
2. Place the leg frame in upright position. Locate the front seat support tube (the front and back seat support tubes are lower than the side support tubes.)
3. Place the front seat z-clips under the front seat support tube.
4. Lower the seat back and snap the back seat clip onto the back support tube.

#### Adjusting Leg Height

Legs are already installed on the bath seat.

To adjust leg height:

1. Press push button and slide leg extension up or down to desired adjustment hole.
  2. Ensure push button fully protrudes through the leg adjustment holes of the leg extension.
- NOTE: An audible “click” will be heard when push button securely protrudes through the adjustment hole.
3. Repeat STEPS 1 and 2 for the other three legs.

NOTE: Ensure all four legs are adjusted to the same height and that all height adjustment buttons protrude fully through the adjustment holes before use.

#### Back Pad Adjustment

The back pad position is adjustable. To change back pad position:

1. Remove the two screws that secure the back pad.
2. Move back pad to desired position.
3. Reinstall the screws. Ensure that back pad is secure before using.



#### Maintenance

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

#### Safety Guidelines – Please Read Before Use

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

#### WARNING

**WARNING:** DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

**WARNING:** Item has a maximum weight capacity of 300 lb. **EVENLY DISTRIBUTED.**

**WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

**WARNING:** ALWAYS inspect the bath seat to ensure that it is properly positioned and stable before using. DO NOT use if bath seat is wobbly or unstable.

**WARNING:** If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately. DO NOT use substitute parts.

**WARNING:** Check leg tips for rips, tears, cracks or wear. If any of these conditions exist, replace leg tips IMMEDIATELY.

**WARNING:** DO NOT use this as a transfer bench or climbing device.

**WARNING:** To avoid falling or tipping: Have someone help you until you know the balance points of your Bath Seat and how to avoid tipping. Do not put downward weight on one side or corner of the Bath Seat. Do not put downward weight on the backrest. Do not reach or lean if you must shift your weight to do so. Ask for help or use a device to extend your reach. Never try to reach an object if you must move forward in your seat and your buttocks do not stay in full contact with the seat. Never lean backward to reach an object.

**WARNING:** All four leg tips **MUST** be in contact with shower/tub floor at all times.

**WARNING:** Ensure that the four chair legs are adjusted to the same height and that the push buttons fully protrude through their respective adjustment holes **BEFORE** use.

**WARNING:** Only use GF Health Products, Inc. accessories with this product.

**WARNING:** This product should only be used with tub floors wider than 18 ¾ inches.

**WARNING:** Users with limited physical capabilities must be supervised or assisted when using the bath seat.

**WARNING:** Make certain ALL screws and nuts are tight at all times.

**WARNING:** Notice for California Customers  
California Proposition 65 **WARNING:** This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

**WARNING:** DO NOT submerge product in water, otherwise damage may result.