



### Features

- Offset handle places user's weight over the base for maximum balance and control
- Ortho-Ease wider grip provides greater support and can be adjusted in angle to minimize stress on the joints.
- Simple height adjustment from 30" – 39" in 1" increments with locking mechanism for added security
- Strong and durable rounded base protects user's legs when walking
- 300 lb weight capacity



GF1100022RevB14

## Silver Collection Low Profile Quad Cane

with the original Ortho-Ease® Grip

Items 6120A, 6121A, 6124A, 6125A



### User Instructions and Warranty Information

Important: Read Before Use



## Instructions for Use – Please Read All Instructions Before Use.

Please consult a healthcare professional for the proper height adjustment.

### Adjusting Cane Height


1. Remove locking knob by twisting counter-clockwise.
2. Adjust cane height: Depress push button and slide lower section upward or downward to desired height.

**NOTE:** An audible "click" will be heard when push button securely protrudes through the adjustment hole.

3. Secure locking knob by reinserting and turning clockwise until tight.

### Adjusting Quad Canes for Right or Left Handed Use

1. Loosen locking knob by turning screw knob counter-clockwise.
2. Press the push button and turn cane handle 180° until push button locks in adjustment hole.
3. Secure locking knob by turning screw knob clockwise until tight. DO NOT over tighten.

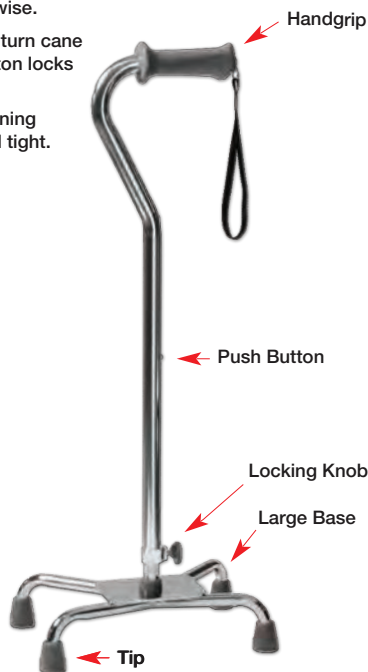

## Maintenance

- Wipe cane weekly with a damp cloth.
- Grips may be cleaned with mild soap or detergent.
- DO NOT use organic solvents, such as acetone, which may damage grip.
- Never use abrasive cleaners or scouring pads on any part of the cane.
- Check locking collet weekly for tightness.
- Replace cane tip as needed.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at

## Safety Guidelines – Please Read Before Use

**WARNING:** DO NOT install or use this equipment without first reading and understanding these instructions. This product should not be used without proper instruction from a healthcare professional.

**WARNING:** Each individual should always consult with their physician or therapist to determine proper adjustment and usage.

**WARNING:** This product has a maximum user weight limitation of 300 lb. The canes are NOT designed to support the total weight of individuals.

**WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

**WARNING:** Before use, ensure cane tip is properly seated and straight. Check rubber tip for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tip immediately.

**WARNING:** Single push button height adjustment MUST be fully engaged and protruding through height adjustment hole on cane to ensure a positive lock. ALWAYS test to see that cane handle is properly and securely locked in place BEFORE using.

**WARNING:** Make sure locking knob is securely tightened at all times.

**WARNING:** When using a Quad cane, the cane should be placed on a level walking surface with all four legs in contact with the surface. Use of cane in a fashion other than this can lead to eventual cane damage and/or personal injury.

**WARNING:** For stability, ensure that shorter legs of the base are closest to foot - otherwise, injury or damage may occur. It may be necessary to adjust cane for right-hand or left-hand use.

**WARNING:** Always use caution when using this product on wet or slippery surfaces.

**WARNING:** If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts.

**WARNING:** Before use, inspect cane and ensure that it is locked securely in position. Inspect weekly.

**WARNING:** Notice for California Customers  
**California Proposition 65 WARNING:** This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

**WARNING:** If any component is not in proper working order or appears worn, contact your dealer for repair and/or appropriate replacement parts.