

# LUMEX®

A Graham-Field Family Brand

## Handheld Shower Holder

Universal Fit  
2 Mounting Options:  
Suction or Screw Installation



### Installation:

- Remove shower holder from mounting bracket.
  - Clean and dry desired mounting area.
  - Place holder on desired location pointing upward.
  - Rotate holder clockwise  $\frac{1}{4}$  turn to lock in place.
- 
- Retira el soporte de la ducha del soporte de montaje.
  - Limpia y seca la superficie de montaje deseada.
  - Coloca el soporte en el lugar deseado, apuntando hacia arriba.
  - Rota el soporte  $\frac{1}{4}$  de vuelta para asegurarlo en su lugar.

***1 Year Warranty***  
***1 Año de Garantía***

### Screw Installation:

- Clean and dry desired mounting area.
  - Use a pencil to mark screw location through mounting base.
  - Remove mounting base. Use a  $\frac{1}{4}$  in. drill bit to gently drill hole at pencil marks.
  - Insert wall anchor. Align mounting base with hole. Assure "top" is oriented upwards. Install screws. **DO NOT OVERTIGHTEN.**
  - Slide shower holder pointing upward onto mounting base. Rotate holder clockwise.
- 
- Limpia y seca la superficie de montaje deseada.
  - Marca la ubicación de los tornillos con un lápiz a través de la base de montaje.
  - Retira la base de montaje. Usa una broca de taladro de  $\frac{1}{4}$  in. para abrir un orificio en la marca del tornillo.
  - Inserta el anclaje de pared. Alinea la base de montaje con el orificio. Asegúrate de que "top" (parte superior) apunte hacia arriba. Coloca el tornillo. **NO APRIETES DESMASIADO.**
  - Desliza el soporte de ducha en la base de montaje, apuntando hacia arriba. Rota el soporte en sentido de las manecillas del reloj para asegurarlo en su lugar.

Manufactured for:  
GF Health Products, Inc.



**graham**  
**field**

**LS**  
Latex  
Statement

**⚠ WARNING:** Notice for California Customers - California Proposition 65  
**WARNING:** This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

\*This product contains natural rubber latex, which may cause allergic reactions