

- ⚠ **WARNING:** Important! Read and understand these instructions before assembling or using your Lumex Drop Arm VersaMode. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the VersaMode is not properly adjusted, personal injury and damage to the VersaMode could result.
- ⚠ **WARNING:** If components are damaged or missing, contact your Graham-Field authorized distributor immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the VersaMode.
- ⚠ **WARNING:** This product has a maximum weight capacity of 300 lb, EVENLY DISTRIBUTED.
- ⚠ **WARNING:** Notice for California Customers- California Proposition 65 **WARNING:** This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.

INTENDED USE

To provide transport, transfer, and commode functions with one device.

ASSEMBLY (SEE PICTURE AT RIGHT)

1. Carefully remove components from box.
2. Install back and seat assembly on base frame assembly.
3. Install right and left knobs at rear seat and tighten until secure — do not overtighten.
4. Install footrests on base frame footrest brackets. Lock footrests into place.
5. Slide the commode pail in under the seat from the back.

- ⚠ **WARNING:** Before use, ensure all components are securely installed and VersaMode is assembled as described above.


OPERATION

Drop Armrests

The VersaMode armrests drop to facilitate transfer. To drop the armrest, pull the release lever outward until the armrest releases, lift the armrest up out of the socket, then swing it downward.

Lock Rear Casters

The rear casters lock into place to prevent movement. To lock, step on the rear pedal until it latches. To unlock, step on the rear pedal until it unlatches.

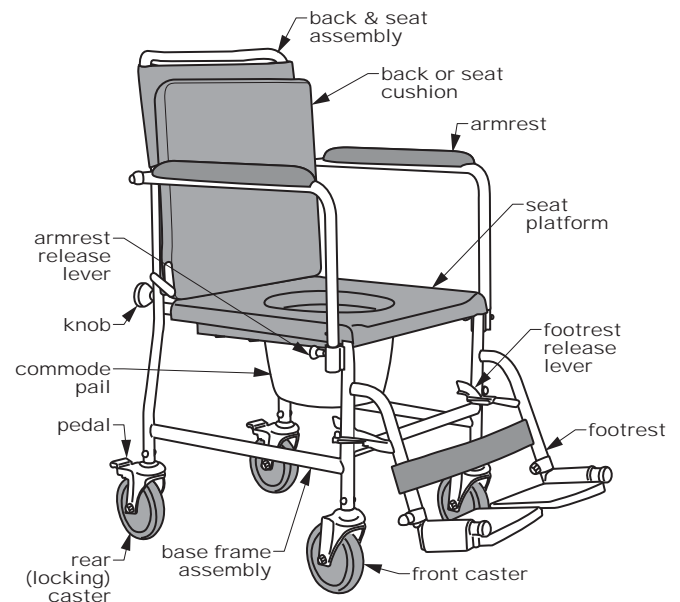

6810A Drop Arm VersaMode

Swing Away / Remove Footrests

The VersaMode footrests swing away or detach to facilitate transfer. To swing the footrest away, depress the footrest release lever and swing it out of the way. To remove the footrest, swing it away, then lift it up off the brackets. To lock the footrest in place, swing it in toward the center of the VersaMode until it locks into place.

MAINTENANCE

Check the fit and tightness of all nuts, bolts and knobs weekly.

⚠ **WARNING:** Inspect the VersaMode weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your Graham-Field authorized distributor for repair and appropriate replacement parts.

Cleaning

Clean the VersaMode with a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure the VersaMode is well-rinsed before use.

- ▲ **NOTICE:** Never use solvents, abrasive cleaners, or scouring pads on any part of the VersaMode.
- ▲ **NOTICE:** Always clean VersaMode as soon as possible after use. To contain odor until VersaMode can be cleaned, replace commode lid on commode pail immediately after use.